## **BuzzClip** Quick Start Guide

For the full instruction manual as well as helpful videos full of tips and tricks, please visit [www.imerciv.com/user-guide](http://www.imerciv.com/user-guide)

# BuzzClip Quick Start Guide

## Getting Familiar with your BuzzClip

Your BuzzClip consists of 4 main parts:

**The Rounded Top (A)** – This is the main control panel for the BuzzClip and it can be found at the top of the device. The sensor is housed here and it slightly protrudes out of it's rounded enclosure. There are two buttons are located to the left and right of the sensor. The rounded top is on a hinged spring, thus this part of the BuzzClip can be rotated and locked at a 90-degree angle so it may also be used as a handheld.

**The Main Body (B)** – The Main Body contains the battery, circuit board as well as the charging port of the device. Edges of the main body have been chamfered, allowing you to run the finger along the part in order to locate the Micro-USB charging port.

**The Lanyard Loop (C)** – This is a small hole or loop located along the chamfered edge of the main body. This hole is to attach the lanyard that is included in your box for additional safety and security purposes (in case of drops, etc.)

**The Vibrator Housing (D)** – This rubberized part is where tactile information is delivered through vibrations. It will always be in contact with your body, so you can easily feel the vibrations. It is located at the back or bottom of the device. The shape of the housing provides a stable anchor point for when the BuzzClip is being worn.

## Operating your BuzzClip

### Turning On your BuzzClip:

**Locate the On/Off Button** – With the BuzzClip in your palm, you should be able to feel a circular ring at the top which is the sensor. To the **left** of the sensor, there is a rubberized button indented with a tactile marking to indicate the **On/Off** button.

1. Press and hold this button to turn on your BuzzClip. You should then immediately feel a series of short vibrations followed by two longer vibrations. The shorter initial vibrations indicate the BuzzClip's battery level:
  - **4 vibrations** indicate that the battery is above **75%**.
  - **3 vibrations** indicate that the battery is between **50% -75%**.
  - **2 vibrations** indicate that the battery is between **25% -50%**.
  - **1 vibration** indicates that the battery is less than 25%. **Charge it soon!**

The two longer vibrations that follow are to let you know that the BuzzClip is ready to go and begin detecting obstacles around you.

2. Your BuzzClip's default starting mode is a **one-meter** range.

## Switching Between Modes on your BuzzClip:

**Locate the Mode Button** – With the BuzzClip in your palm, you should be able to feel a circular ring at the top which is the sensor. To the **right** of the sensor, there is a rubberized button with NO tactile mark. This is the **Mode** button.

1. You can only switch between modes when the BuzzClip is **ON**. The device comes with three-modes: a **1-meter** mode, a **2-meter** mode and a **3-meter** mode. We suggest using the 1-meter mode for indoor use and the 2-meter and 3-meter modes for outdoor use.
2. While the BuzzClip is **ON**, Press and hold the **Mode** button to change modes on your BuzzClip. You should then immediately feel **two short rapid vibrations**. This indicates that the BuzzClip is now in the **2-meter mode**.
3. Press and hold the Mode button again and you should then feel **three short rapid vibrations**. This indicates that the BuzzClip is now in the **3-meter mode**. We recommend using this mode for open areas and general landscaping of the surrounding environment.

## Turning Off your BuzzClip:

**Locate the On/Off Button** – With the BuzzClip in your palm, you should be able to feel a circular ring at the top which is the sensor. To the **left** of the sensor, there is a rubberized button indented with a tactile marking to indicate the **On/Off** button.

1. Press and hold this button to turn **off** your BuzzClip. You should then immediately feel one long continuous vibration. Your BuzzClip is now **Off!**

## Wearing your BuzzClip

The rounded top where the sensor and buttons are housed can be slightly pulled away or rotated from the main body due to the hinged spring. Your clothes are to go in between this space that separates the rounded top from the main body of the BuzzClip.

## Fastening the BuzzClip:

1. The main body and the vibrator housing always go under or between your clothing. Only the rounded top with the sensor and the buttons should be exposed on the outside of your clothing (i.e. on the lapel of a jacket, between the buttons on a shirt or blouse, on the rounded neck of a T-shirt, on the beltline, out of pockets, on the sleeves of a shirt, etc.)
2. Always ensure that the sensor is exposed outwards and is not covered by anything so that you do not receive any false positive responses.
3. Ensure that the sensor is as parallel to your body as possible. For example, if you are wearing a buttoned up shirt, make sure that the sensor is parallel to your chest and that it is not tilted to the left or right.
4. Ensure that you are able to operate both buttons on the BuzzClip from where you are wearing it.

5. Ensure that the vibrator housing on the back of the device is pressed up against you and in proper contact with your body so that you always feel the vibrational information delivered to you.

The BuzzClip can be worn many different ways based on one's mobility needs. It can be easily clipped onto shirts, t-shirts, on sleeves (long or short), sweaters, jeans, sweatpants, and more. Multiple BuzzClips can also be used at once for greater coverage around one's body. This would give a blind or partially sighted user much more information about their immediate surroundings. For example, one on the chest and one of each sleeve in order to protect your front and sides.

## Holding your BuzzClip

Your BuzzClip can be used as both a wearable and a handheld!

### Locked Position:

1. Place the device in your hand with the vibrator housing in your palm.
2. Position or slide your thumb under the rounded top and you will be able to push the sensor up until it locks at a 90-degree angle so that the sensor is now facing forward rather than up at you.
3. You can now hold the BuzzClip in your hand similarly to how you would hold a flashlight and use it as a handheld to detect obstacles. We recommend using the 3-meter mode when holding the BuzzClip.

## Unlocked Position:

1. While in the locked position (sensor at 90 degrees), between the rounded top and the main body, there is a coiled spring. Feel along this spring and locate the plastic stopper on the right side.
2. Gently push the rounded top a little past a 90-degree angle and then slide this plastic stopper to the right with your thumb.
3. Now the BuzzClip is in the Unlocked Position with the rounded top and sensor back in their resting states.


## Using your BuzzClip

The BuzzClip is not a replacement for a cane but a complementary mobility tool. It is intended to be used in combination with a cane or a guide dog. For those who frequently travel with a cane or a guide dog, we recommend using the BuzzClip to address upper body and head level obstacles while the cane can address changes in elevation and lower obstacles. The BuzzClip can also be used on its own for when in familiar indoor environments and for people living with partial sight who do not use a cane or a guide dog.

### **Detection Accuracy and Range of the BuzzClip:**

1. The BuzzClip can accurately detect anything coming into its range of detection. The sensor ensures that the distance to any nearby obstacles is accurately measured regardless of lighting conditions or transparency of the object. It does this by sending and receiving sound waves. The received information is then relayed back to the user instantly through intuitive vibrations.

2. The angle of detection is a conical shape from the center of the sensor (mesh within circular ring) and the BuzzClip has three ranges of detection: a 1-meter mode, a 2-meter mode and a 3-meter mode.

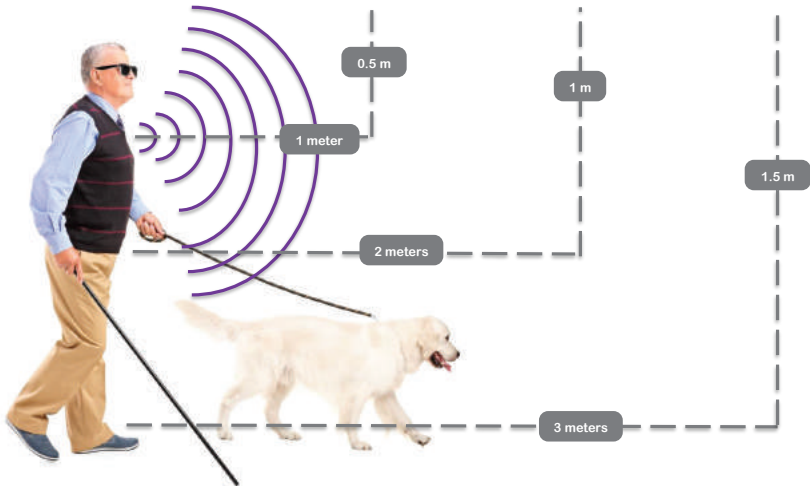3. When the BuzzClip is clipped around your upper body or chest area in the **2-meter** mode, the detection range will cover you shoulder to shoulder and from about your waist to just above the top of your head.
4. When the BuzzClip is clipped around your upper body or chest area in the **3-meter** mode, the detection range will cover you more than shoulder to shoulder and from about your ankles to just above the top of your head.
5. The frequency of vibrations from the BuzzClip increases as you get closer to an obstacle.
6. The BuzzClip delivers three strong rapid vibrations when it is 50 cm (arm's length) away from an obstacle as an indication to stop and proceed with caution as there is something that is now less than 50 cm away from you.

### **The Automatic Sleep Feature:**

1. The BuzzClip has a built-in automatic sleep feature to conserve battery and to reduce irritation for when remaining a fixed distance away from a person or obstacle.

2. For example, if you were having a conversation with someone face to face, you would be within 1 or 3 meters of each other, meaning the BuzzClip would be vibrating constantly while you are conversing.
3. The automatic sleep feature activates when you are a fixed distance away from a person/object for longer than **5-7 seconds**.
4. After 5 to 7 seconds of being a fixed distance away from something (i.e. you are not moving and neither is the obstacle/person), the BuzzClip will stop vibrating and is now in its 'sleep mode'.
5. As soon as you or the person/object begins to move more than 5 cm, the BuzzClip will kick back into gear and start detecting and vibrating again.

## Using your BuzzClip in Wet Winter Weather:

1. The BuzzClip is water resistant. It will function perfectly in light rain. Don't go swimming with it though!
2. If rain drops land directly into the sensor, the rain may create a film of water in the sensor's mesh. The unit may stop functioning temporarily. Simply wipe or shake the water off or out of the sensor, give it a few minutes to recalibrate and then it's good to go!

3. The BuzzClip can function in extreme winter conditions (-30°C)
4. Ensure that the sensor is always exposed even with heavy winter clothing.

## Charging your BuzzClip

The BuzzClip's battery will easily last up to 10 hours under rigorous use and can be charged using a standard Micro-USB cable.

## To Charge your BuzzClip:

1. Plug in the provided Charger into an electrical outlet in your home.
2. Plug in the larger end (USB) of the provided charging cable into the Charger.
3. Locate the Micro-USB Charging Port on the main body of the BuzzClip.
4. Plug in the smaller end (Micro-USB) of the charging cable into the Micro-USB charging port on the BuzzClip.
5. Your BuzzClip is now charging! It should take less than an hour for the BuzzClip to reach a full charge. The BuzzClip indicates its battery level to you through specific vibrations as soon as it's turned on:
  - **4 vibrations** indicate that the battery is above **75%**.
  - **3 vibrations** indicate that the battery is between **50% -75%**.
  - **2 vibrations** indicate that the battery is between **25% -50%**.
  - **1 vibration** indicates that the battery is less than **25%**. **Charge it soon!**

## Connecting the Lanyard to your BuzzClip

**Locate the Lanyard Loop** – This is a small hole or loop located along the chamfered edge of the main body. This hole is to attach the lanyard that is included in your box for additional safety and security purposes (in case of drops, etc.)

### **To Attach the Lanyard:**

1. Locate the thin rope that is attached to the small plastic adaptor.
2. Thread the thin rope through the lanyard loop.
3. Open the end of the rope that has been threaded through the lanyard loop and pull the other end of the lanyard all the way through to firmly attach the lanyard to the device.
4. To detach the lanyard, push the small round button at the center of the plastic adaptor and pull out the ring on the perimeter of the adaptor.

## Disclaimers and Warranty Information

Warning: Please read this document before using the device. Follow the instructions listed below to operate the device safely.

### 1.0 PRODUCT DISCLAIMER

- Read the instruction manual carefully before use.
- Do not use the BuzzClip while it is charging.
- Do not wash your BuzzClip in the washing machine.
- Do not wash your BuzzClip with water.
- Do not clean it when the BuzzClip is turned on or being charged.
- Do not clean the BuzzClip with household cleaning products.
- Do not swim with your BuzzClip.
- Do not take showers with your BuzzClip.
- Do not dry it inside the microwave or drier.
- Do not expose the BuzzClip to temperatures above + 70°C.

- Do not use it under -30°C.
- Do not clip the BuzzClip onto your skin or your hair.
- Do not open the BuzzClip.
- Do not remove or replace parts.
- Do not leave it exposed to heat for long periods of time.
- Do not sink it into any liquid.
- Do not ingest parts of the BuzzClip.

### 1.1 Warning:

As with many other electrical devices, failure to take the following precautions can result in serious injury or death from electric shock, fire, or damage to the BuzzClip.

- Use only an approved charging cable
- Do not allow a wireless device or charging cable to become wet
- Unplug the charging cable during lightning storms or when unused for long periods
- To prevent damage, avoid sitting or stepping on a wireless device, its cable, or the cable plug or connector.

## 1.2 Cables and Cords:

To reduce potential trip hazards or entanglement hazards, arrange any cables and cords so that people and pets are not likely to trip over or accidentally pull on them as they move around or walk through the area and do not allow children to play with cables and cords.

## 1.3 Charging

- Always use the charger provided with the device. The charger has the shape similar to a regular mobile phone charger and is placed in the packaging box adjacent to the device.
- Use the charger only in dry locations. Keep away from rain, snow or excessive moisture.
- It is recommended that the battery is charged before reaching the deep discharge state.

## 1.4 Battery Safety:

- Do not crush, open, puncture, mutilate, heat above 70°C, apply direct heat to, or dispose of batteries in fire.
- Do not leave a battery-powered device in direct sunlight for an extended period, such as on the dash of car during the summer.

- Do not connect batteries directly to wall outlets or car cigarette-lighter sockets.
- Do not attempt to disassemble or modify batteries in any way
- Do not recharge batteries near a fire or in extremely hot conditions

## 1.5 Warning: The Buzzclip is water resistance but not water proof.

- To reduce the risk of fire or shock, do not submerge the device under water or any other liquid.

## 1.6 Use in accordance with these instructions:

- Do not use the BuzzClip near any heat sources.
- Clean only with a dry cloth
- Unplug before cleaning

## 1.7 Recommendations

- Store the BuzzClip in a cool dry place.
- Clip the BuzzClip on your clothes only.

- Adjust the lanyard to be comfortable and not tight on the neck.

- Clean the outside of the buzzclip with a wet cloth. While cleaning, avoid spilling water on the sensor. Dry it with a clean and dry cloth.

## 2.0 Precautions in using the BuzzClip

Important precautions for user of the BuzzClip:

- Using the device effectively in day to day mobility requires regular practice and usage.

- The BuzzClip is not meant to replace white cane or guide dog.

- The device should be used only after setting the sensor at a correct angle. The sensor direction must point towards the chest of a person standing in front at a distance of 2 meters.

- Before starting a journey, check if the device is charged. Please ensure that the device is charged periodically every 1-2 days. More frequent charging may be required if the usage is more than 5 hours. Once the battery is fully charged, the device is expected to function for at least 10 hours under normal usage.

- Please ensure proper functioning by placing your hand in front of the sensors, thereby triggering pattern four

vibrations. Set the MODE switch correctly depending upon whether you are indoors or outdoors.

- Please use the device only when the sensor is dry. If the sensor is wet due to rain or water spillage, the device might temporarily stop functioning or might function inaccurately. As in the case of most cell phones, please do not expose the device to water or wet environments. The device is “splash proof” and can withstand light spillage or splashes of water or other liquids. Allow the water in the sensor area to dry before checking and using the device.

- Never submerge the device in water or liquid, this may cause permanent damage to the device.

- Do not short circuit the charging point as it can damage the device and might lead to fire.

- While using the device in crowded places, the short range mode is recommended. This will reduce the detection range to 1 meter in front of you. However, if you continue operating in the long range mode, the device would continuously vibrate due to the numerous objects / people within 2 meters.

- There may be an initial adaptation phase while learning the use of the device. During this time your walking speed might reduce slightly due to increased focus on vibrations. However, with regular practice, you are likely to get accustomed to these vibratory patterns and be able to walk at your regular or increased speed.

- The detection of fast moving obstacles is done within the range of 2 meters only. This feature is intended only as a cautionary alert and you may not be able to fully move out of the way.
- The visually impaired USER is the intended OPERATOR.

**The BuzzClip needs to be used in accordance with the information provided in this manual.**

### **3.0 Maintenance and repairs**

No modification of this equipment is allowed. In case repairs or servicing are required, please contact the concerned service personnel.

- Do not attempt to expose or remove the internal components of this device as this may expose you to risks arising from the electronic or mechanical components.
- Do not short circuit the charging point as it can damage the device and might lead to fire.
- Do not attempt to clean, service or conduct maintenance while the device is charging.
- Do not attempt to replace the battery yourself. If required please contact the concerned service personnel.
- Do not put this device in fire, as there could be an explosion hazard due to the presence of the battery.

- Do not attempt to take apart, open, service, or modify the BuzzClip. Doing so could present the risk of electric shock, damage, fire, or other hazard. Any evidence of any attempt to open and/or modify this device, including any peeling, puncturing, or removal of any of the labels, will void the Limited Warranty and render the BuzzClip ineligible for authorized repair.

For repair Services, Please contact the retailer that you purchase the Buzzclip from.

### **4.0 Warranty**

Thank you for your interest in the products and services of iMerciv Inc.

This Limited Warranty applies to physical goods, and only for physical goods, purchased from iMerciv Inc.

#### **4.1 What does this limited warranty cover?**

This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. The warranty will cover any products that proved defective because of improper material or if the customer received the product broken.

During the Warranty Period, iMerciv Inc. will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.

#### 4.2 What will we do to correct problems?

iMerciv Inc. will either repair the Product at no charge, using new or refurbished replacement parts.

#### 4.3 How long does the coverage last?

The Warranty Period for Physical Goods Purchased from iMerciv Inc. is 365 days from the date of purchase.

A replacement Physical Good or part assumes the remaining warranty of the original Physical Good or 30 days from the date of replacement or repair, whichever is longer.

#### 4.4 What does this limited warranty not cover?

The Limited Warranty does not cover any problem that is caused by:

- Conditions, malfunctions or damage not resulting from defects in material or workmanship.
- Damage resulting from improper maintenance or unauthorized modification of the product
- Damage resulting from natural disasters
- Theft or loss of a product
- Damage of a product resulting from negligence

#### 4.5 What do you have to do?

To obtain warranty service, you must first contact us to determine the problem and the most appropriate solution for you. Contact us at [info@imerciv.com](mailto:info@imerciv.com).

### 5.0 Production Specifications

#### 5.1 Physical Specifications

Dimensions (H x W x D)	: (45 x 35 x 65) mm
Sensor-angle	: 30 degrees
Device Weight with Battery	: 62g
Enclosure Material	: Aluminum

#### 5.2 Environmental Specifications

Operating Temperature	: -30° to +70° C
Storage Temperature	: -30° to +70° C
Relative Humidity Range	: 15% to 93% (RH)
Storage Humidity Range	: 0% to 95% (RH)
Barometric Pressure Range	: 700 to 1060 hPa
Storage Pressure Range	: 500 to 1060 hPa

### 5.3 Battery Specifications

Type : Rechargeable Li-polymer

Rating : 3.7 V; 350mAh

Size : (50 x 20 x 3.5) mm

Weight : 8 grams

Operating Time : Approximately 10 hrs

Charging Time : 2 hours

### 5.4 Charger Specifications

Type : Travel charger, certified

Input rating : 100-240 V AC~, 50/60 Hz, 0.5A

Output rating : 5 V DC, 0.5 A

### 5.5 Detection Characteristics

Sensor Type : 41 kHz Ultrasonic

Maximum Angular Coverage : 30°

Resolution : 1 cm at 2 m

Indoor Range : 0.3 to 1.0 m

Outdoor Range : 0.3 to 2.0 m

### 5.6 Service Life

The expected service life of this product is five years, if it is maintained and used as per the manufacturer's instructions.

### 5.7 Others

No significant effects of ultrasound have been identified on pets or pests. No significant effects of sun light, dust or water have been observed on the sensor.

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