



TR 2000/3000

Shower Trolleys

*Hygiene Equipment
Designed for People*

TR 2000/3000, makes both showering and transfers easier for attendants

The TR 2000 hydraulic and TR 3000 battery operated shower trolley is unique with its side mounted column, allowing the trolley stretcher to overlap the bed further for a safe and easy resident transfer. Both models available in 3 different lengths.

CE



Registered at Medical Products Agency, Sweden, as a MD Class 1 product, MDD 93/42EEC.

TR reserves the right to make design changes and/or improvements without notice.



The trolley assists and functions already at the bed side allowing the attendant to have easy access to the trolley controls including height adjustment, stretcher tilt, castor brakes and straight steering. The trolley stretcher has drains at both ends allowing the person to be positioned in either direction. There is no head or foot end specified.



TR EQUIPMENT



TR 2000/3000 Shower Trolleys

Hygiene Equipment
Designed for People

Features

- Foot operated and height adjustable enables the attendant to find the most efficient and convenient working height and gives the attendant both hands free to tend to the patient.
- Thanks to the belt driven high/low device on the TR 3000 it operates quietly, smoothly and free from sudden movements. The height adjustment is driven by two high capacity rechargeable batteries and when charging is needed an audible tone sounds.
- The stretcher can easily be tilted to the "Trendelenburg" position, to alleviate person in shock. The stretcher can move in either direction with the tilt control handle located at both ends. The tilt position is controlled by gas springs so that the attendant has infinite tilt positions within the tilt range.
- The large castors with brakes and low angled legs with straight steering castors are features that make the trolley easy to steer and maneuver.
- The low center of gravity of the chassis lends stability to the trolley.
- The side rails can be pressed together when traveling through narrow doorways reducing the trolley width.
- The chassis is made of powder coated stainless and zinc plated steel.
- Designed for easy cleaning.



TR 3000 Battery operated - Back of chassis



TR 2000 Hydraulic - Back of chassis



TR 2000 Hydraulic - High chassis and
TR 3000 Battery operated - High chassis

Specifications



Length	1600/1900/2100 mm (63"/75"/83")
Width	705 mm (27 3/4")
Height	625-925 mm (24 1/2" - 36 1/2"), <i>Standard</i> 685-985 mm (27" - 39"), <i>High chassis, option</i>
Working Load Limit	150 kg (330 lbs)
Chassis	Powder coated stainless and zinc plated steel
Castors	Two castors with brakes and two with straight steering
Control	Foot control for height adjustment
Mattress, Pillow	Flame retardant material - Grey (Standard)/Yellow or Blue (Option)
Battery (TR 3000)	Rechargeable, 24 V

Models - Hydraulic

TR 2000	Standard	L = 1900 mm
TR 2000 - S	Pediatric	L = 1600 mm
TR 2000 - L	Long	L = 2100 mm


Models - Battery Operated

TR 3000	Standard	L = 1900 mm
TR 3000 - S	Pediatric	L = 1600 mm
TR 3000 - L	Long	L = 2100 mm

Accessories/Options

- Backrest wedge pillow
- For slinglift transfers, a factory mounted set of castors allowing a minimum of 135 mm free space under the front legs can be added. It raises the trolley stretcher 60 mm.
- All models can be delivered with blue or yellow mattress and pillow.

Dimensions (mm)

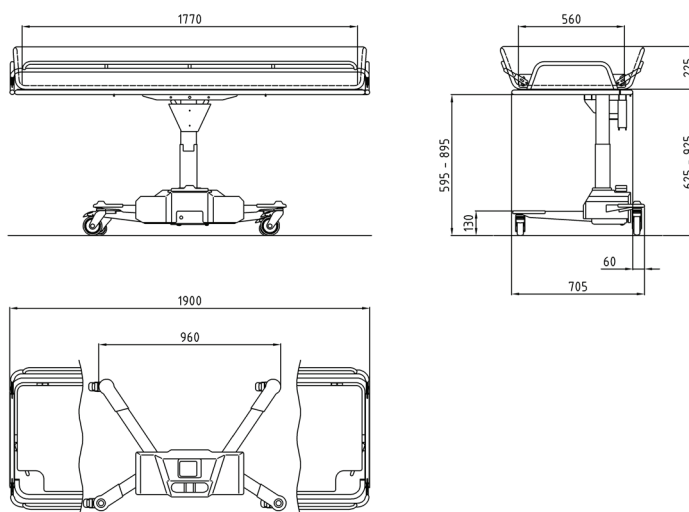

Head office

TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT