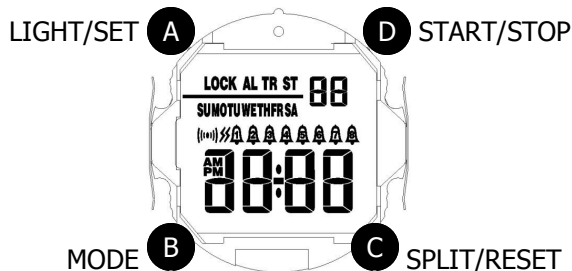


WobL™

"The World's Smallest Vibrating Watch"

Instructions

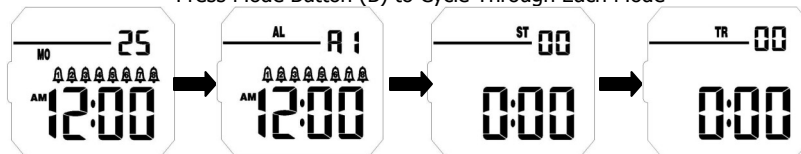


Watch Functions

Time display in month, day, date, hour, minute, second
8 sound or vibration alarms (each alarm is 15 seconds)
Stopwatch (chronograph) 1/110 second
Countdown timer (range 30 min. to 24 hr.)
Time limit - Set the time range for alarms in Military time
Lock function - Lock buttons to prevent accidental re-programming
12/24 hour display
Display light

Function Modes

Press Mode Button (B) to Cycle Through Each Mode



Time Mode

Alarm Mode

Stopwatch Mode

Timer Mode

TIME LIMITS ARE IN MILITARY TIME



NOT WATERPROOF

To Set Time, Date, & Month

Press B - To advance to *Time Mode*
Press and hold A until the seconds flash
Press B - Minutes will be flashing
Press D - To advance the minutes
Press B - Hour will be flashing
Press D - To advance hour
Press B - Date will be flashing
Press D - To advance date
Press B - Month will be flashing
Press D to advance month
Press B - Year will be flashing
Press D to advance year
Press A to confirm time and date settings

Press D to change 12/24 hour display

Press C to display the date

To Set Alarm/Vibration

Press B - To advance to *Alarm Mode*
Press D - To turn on the alarm or vibration
When sound alarm is on (■■■■) will display
When vibration is on <<<<< will display

Vibration **AND alarm cannot be turned on at the same time.

Each vibration or alarm is 15 seconds.

To Set 8 Individual Alarms

Press B - To advance to *Alarm Mode*
Each alarm is numbered A1-A8
To set Alarm 1, press and hold A until hour flashes
Press D to advance the hour
Press B - Minutes will be flashing
Press D to advance the minutes
Press A to confirm the time entered



Will appear to show the alarm has been set

Press C to select alarm A1-A8
Continue above steps to set additional alarms
Press B - To return to *Time Mode*

To Set Countdown Timer

Press B - To advance to *Timer Mode*
To set time limit (ex. Alarm from 0700 to 2000)
Press A and hold until starting hour flashes
Press D to advance the hour
Press B - Ending hour will be flashing
Press D to advance the hour
To set time interval (ex. Alarm every hour or every 2 hours)
Press B - Hour will be flashing
Press D to advance the hour
Press B - Minutes will be flashing
Press D to advance the minutes
Press A to confirm the timer setting
Press D to start the countdown timer
Press B to return to *Time Mode*
To Stop countdown timer - Press D

Must restart timer daily when using time limits


To Lock Buttons

Press B - To advance to *Time Mode*
Press B and C together at the same time to lock the buttons
"Lock" icon will appear
When buttons are locked only light button will be working
Press B and C together at the same time to remove the lock-out feature ("lock" icon will disappear)

To Set Stopwatch

Press B - To advance to *Stopwatch Mode*
Press D - To start the stopwatch
Press D again to stop
Press C to return to zero

To Erase An Alarm

Press B to advance to *Alarm Mode*
Press C until you get to specific alarm you want to erase
Press and hold C until  disappears
Press B to return to *Time Mode*

WobL

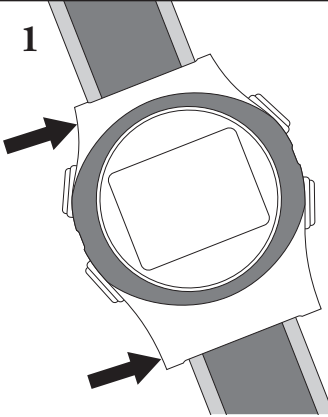
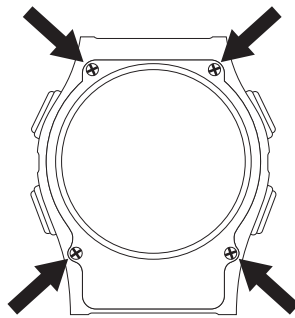
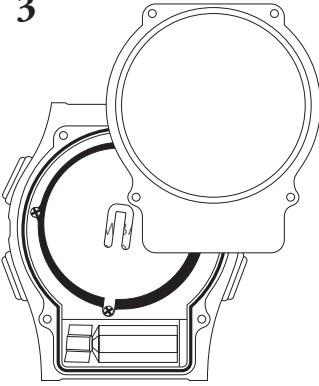
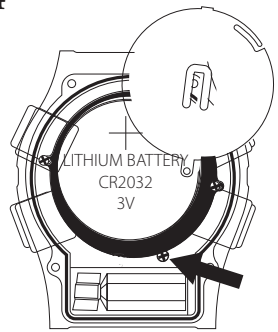
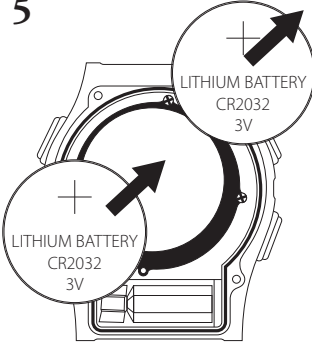
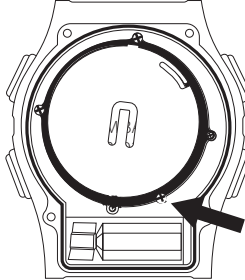
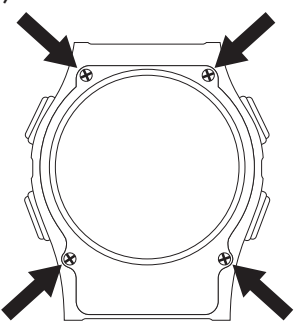
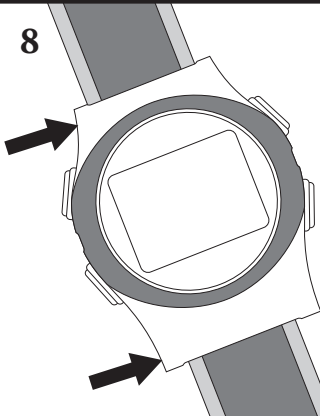
Limited Warranty

Your watch has been manufactured to high standards and all parts have been assembled and tested with great care. Your watch is guaranteed against all defects for a period of 6 months from the date of purchase. The guarantee however does not cover damage through accident, negligence or misuse. This guarantee does not affect your statutory rights. **PottyMD IS NOT LIABLE FOR ANY SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES.** IMPORTANT – not covered by guarantee: 1. Batteries, straps and display screen. 2. Damage caused by accident, misuse, incorrect battery replacement or lack of care. Your watch is water resistant (3 ATM) and will withstand accidental splashing or rain but is not suitable for swimming. The watch should not be operated near water. If it is determined that your watch has been damaged by water it will void the warranty. If your watch requires attention under the terms of the warranty then please mail the watch only (we suggest certified or insured delivery) along with proof of purchase and a brief note to explain the fault to the address below. Do not send the box or other packaging material if you are claiming a faulty watch. If the watch is determined to be faulty within 6 months of purchase then PottyMD LLC will replace the watch free of charge. PottyMD will not refund shipping or any other expenses. If the watch is determined to be functional and without defect then it will be returned to the customer. PottyMD LLC will not refund monies for any watch that is worn or shows any wear or damage. In the event that PottyMD LLC elects to return monies for an unused watch, the customer may be charged a 20% restocking and processing fee.

WobL Battery Instructions

Please Read Carefully

Battery life depends on usage (vibrating watches require frequent battery changes). **Battery type is CR 2032.** Keep batteries away from children. Please contact PottyMD if you have any questions regarding the battery replacement process.

<p>1</p>  <p>Remove the band by pushing the spring bar out of the holes.</p>	<p>2</p>  <p>Unscrew the four screws at the case back. Keep the screws in a safe place.</p>	<p>3</p>  <p>Remove the case back out of the watch case.</p>	<p>4</p>  <p>Unscrew the screw on the metal clip and remove the metal clip.</p>
<p>5</p>  <p>Replace old battery and insert new battery with positive (+) side up.</p>	<p>6</p>  <p>Replace metal clip, screw into place. Make sure rubber seal stays in place.</p>	<p>7</p>  <p>Replace the screws.</p>	<p>8</p>  <p>Replace the band to the watch case by fixing the spring bars into the holes.</p>