





# Model 14

The Model 14 was designed to offer additional postural support to the wheelchair user.

Available in child and adult sizes. This model has the advantage of being **sold in separate components**.

Depending upon the individual requirements Model 14 can start from a simple lap belt and be transformed into a full harness by adding shoulder straps and/or a groin strap as and when required.

Consequently the shoulder straps are unfixed to the lap belt and detachable unlike the B78 harness below.

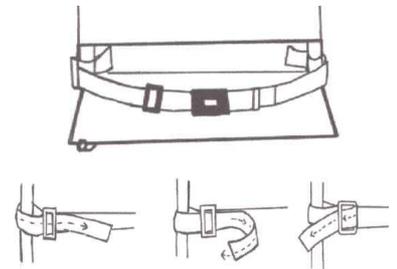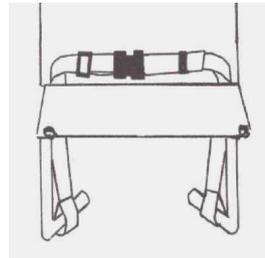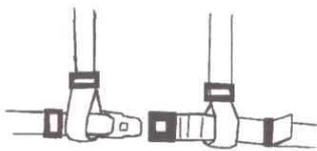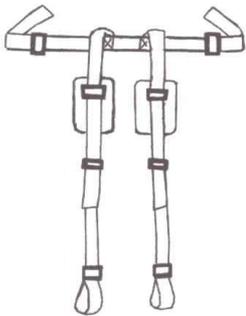

The Shoulder Straps 'slide-fix' to the tubular handles of the wheelchair & loops on to each side of buckle

On a **manual wheelchair** the lap belt loops over the tipping levers

On a **Power Chair** the lap belt 'slide-fixes' to the tubular framework.

Waistbelt available in either '**Loop-on**' for manual Wheelchairs or '**Slide-fix**' for Power Chairs.

**Choice of buckles available.**



Sea Sure Lever Buckle



Side Squeeze Buckle



Press Button Vehicle Buckle



Steel Safety Buckle

## Model B78

(An upgraded version of Model 14)

Full supportive body harness complete with understrap which fixes to chassis cross-section.

The shoulder straps are sewn into fixed, position on lap belt for extra stability.


Adjusts at each side of lap belt buckle for ease of use.

**Fitted with Vehicle Buckle as standard. One size only.**

Two types available:

**'Loop-on'** for manual Wheelchairs

**'Slide-fix'** for Power Chairs.

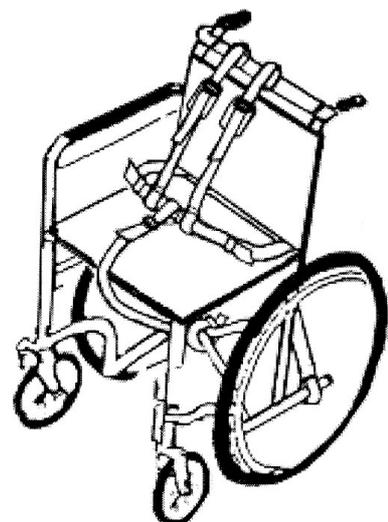

# Model 14 & B78 Fitting Instructions

**Shoulder strap adjusters.**  
To loosen or tighten the straps.

The **shoulder straps** 'slide-fix' onto the push handles & then pass down to loop onto the lap belt at either side of the central buckle.

**To lengthen straps:**  
Pull the adjuster upwards to release the webbing.

**To shorten straps:**  
Pull the tail-end of the strap through the adjusters.

The black plastic 3-bar '**Sliplok**' slides hold the straps more securely into position.


**Lap belt** loops onto the 'tipping levers' of manual chairs.

For **power chair** belt see 'slide-fix' method below.





The **groin strap** is fixed around the cross-section of the framework underneath the wheelchair seat by using the 'slide-fix' method. (See instructions below).

The **groin strap** loops over the central lap belt buckle.


Adjust the length by pulling the tail-end of the strap through the black plastic adjuster buckle.

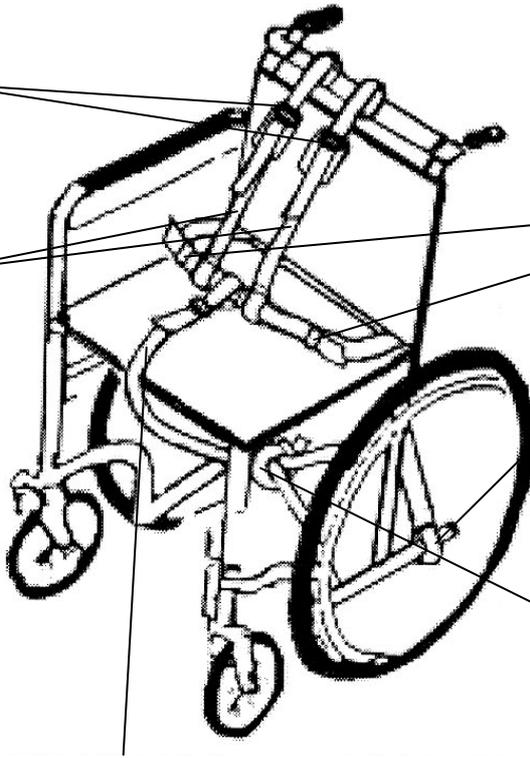

## Slide-Fix Method

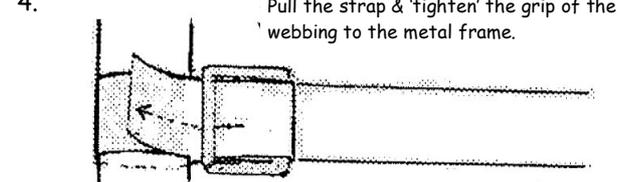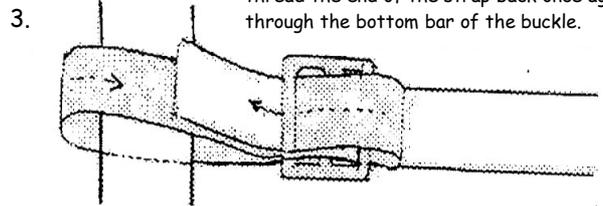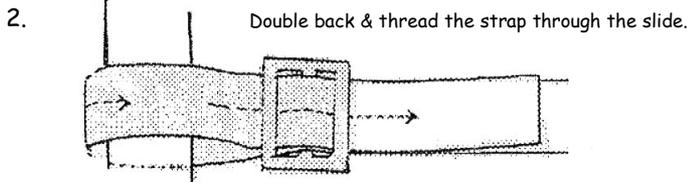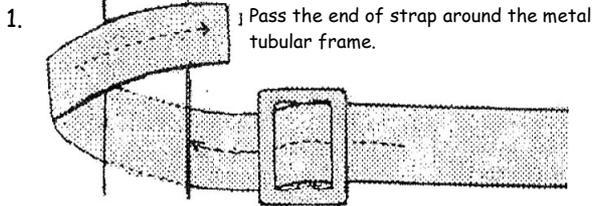


Sea Sure Lever  
(19.60) 50mm Buckle  
B/L 560 Lbs / 200 Kg

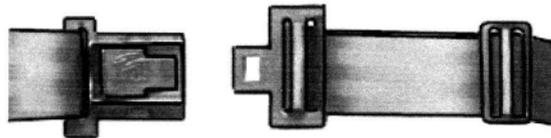

Press lever/thumb-catch to release

Sliplock slide added to prevent slippage

Duraflex Side Squeeze Buckle  
B/L 220 - 208 Lbs  
25mm, 40mm & 50mm  
(Not suitable for overweight patients)

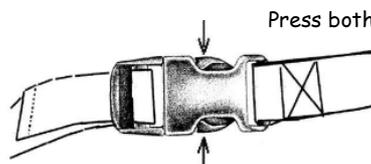

Press both sides simultaneously to release

Adult Vehicle 50mm  
Press Button Buckle  
B/L 6000 Lbs.

