
Safety Step Stool

INSTRUCTION MANUAL

**KEEP YOUR
BALANCE
WHEN
STEPPING
UP AND
DOWN**



Model No. EL5804

Please read carefully before use and keep for future reference

Customer service: 0844 441 8448

For more innovative products from EasyLife please visit:
www.easylifegroup.com

Dear Customer

Thank you for purchasing the Safety Step Stool.

Getting in and out of the bath, reaching top shelves, cleaning windows, changing light bulbs, there are lots of reasons why this Safety Step Stool is such a smart buy. With a useful 12" (28cm) high step, it features a stability handrail for reassuring support and peace of mind. Lightweight and portable yet exceptionally steady and stable, it's made from sturdy steel, with cushioned comfort hand grip and features a non-slip step surface for sure-footed confidence.

PARTS

- 1 x Handle
- 4 x Locking nuts
- 4 x Bolts
- 1 x Step
- 2 x Uprights

TOOLS REQUIRED

PLEASE NOTE: For your safety this product uses locking nuts. They cannot be tightened by hand and you should use the tools recommended below to tighten them.

- 10mm spanner (not supplied)
- Crosshead screwdriver (not supplied)

ASSEMBLY

- Using a spanner and screwdriver attach the uprights to the step using 4 x bolts and 4 x locking nuts (as shown in fig 1). Please note it is recommended to only tighten the nuts / bolts loosely, do not over tighten at this stage.
- Insert the handle into the uprights and secure in place so the two parts click into place.
- Fully tighten the nuts / bolts.


WARNING

- Maximum weight load 150kg / 23st 8lbs.
- Do not exceed this weight load limit.
- Do not use near hot surfaces, cookers or areas where you could potential harm yourself if you fall.
- Do not stand on the handle at any time.
- Do not jump on and off the step.
- Do not use on uneven or unstable surfaces.

DISPOSAL

- Please respect the environment!
- Dispose of the packaging according to the local recycling regulations.
- Dispose of the appliance at the end of its life span according to environmental considerations.

FIG.1

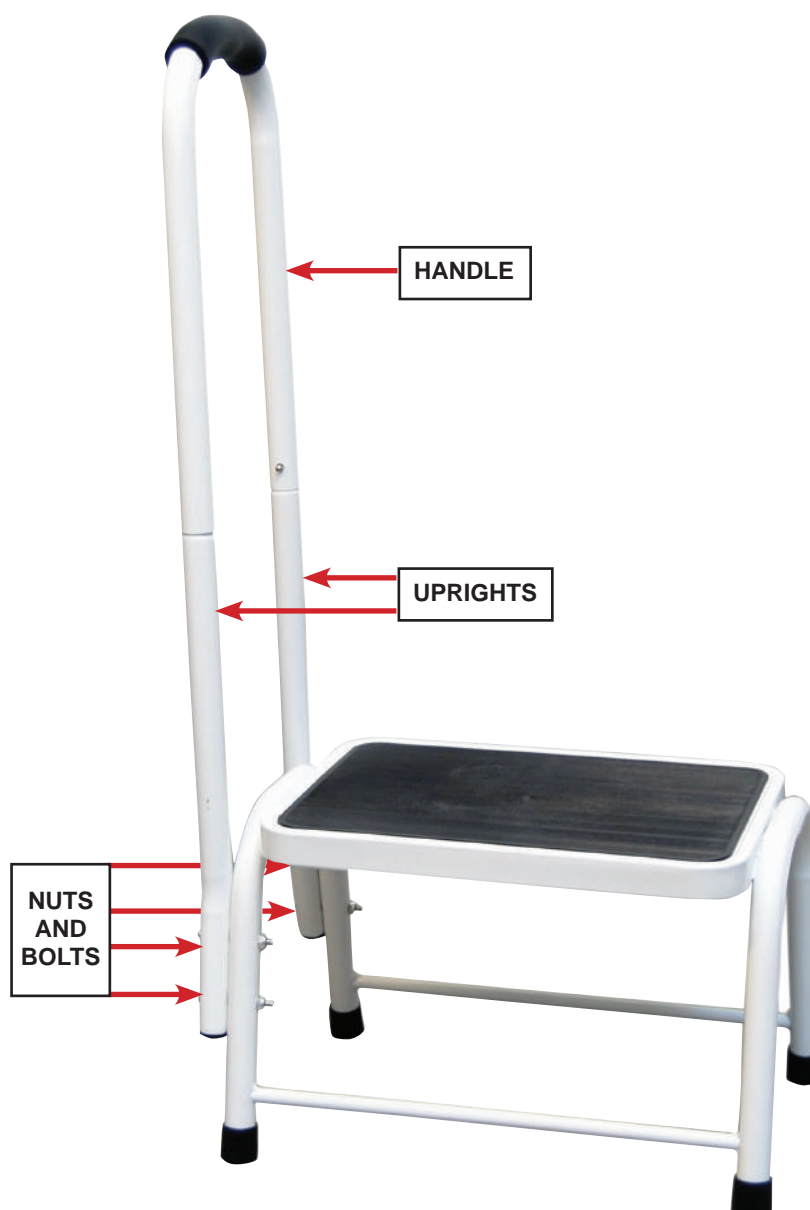

⚠ WARNING

This product is intended for domestic use only. Maximum weight load 150kg / 23st 8lbs.

TECHNICAL SPECIFICATION

Measurements: Step: H26cmxW38cmxD28cm (H10¹/₄xW15xD11in). Handrail: H89cm (H35in). Weight: 3kg (6.7lb)

Imported and distributed by EasyLife Group Ltd, London NW1 0JP

Customer service: 0844 441 8448

For more innovative products from EasyLife please visit:

www.easylifegroup.com

INS804-1 ©2015 EasyLife Group Ltd

 **easylife**TM
Est. 1992 everyday solutions