



SmitCare Limited
14 Camberley Business Centre, Bracebridge
Camberley, Surrey GU15 3DP
Tel: 01276 677888 Fax: 01276 691565
Email:sales@smitcare.co.uk



Instructions for the safe fitting and use of a Boomerang Board **Code STBTB**

Manufactured in China for SmitCare
Maximum user weight 190kg (30 stone)



The SmitCare Boomerang Board is a moving and handling device used by people with limited mobility to transfer from one sitting position to another. The type and extent of mobility impairment may dictate the best method of use for the board, so it is advisable to consult with a trained health professional such as a Moving and Handling Advisor or Occupational Therapist before first use.

The following points should be noted when using the Boomerang Transfer Board:

- 1) Make sure the surfaces you are transferring between are as close as possible and ideally at a similar height.
- 2) Make sure that the Boomerang Board is the correct way up. The non-slip rubber pads must be on the underside to prevent movement of the board during the transfer.
- 3) When transferring to or from a mobile surface, such as a wheelchair or glideabout, ensure that the brakes are applied.
- 4) If possible, having achieved the transfer, place the Boomerang Board in a position where it can be easily reached so that the return transfer can be affected safely.

Questions / comments? Please contact our Customer Services department
on Lo-call : **0845 1303 747**