



# User Instructions

## NRS Wheeled Commode

L22056	NRS Wheeled Commode Fixed Height		Max User Weight
M99448	NRS Wheeled Commode Wide/Fixed Height		
N54411	NRS Economy Wheeled Commode		
M66119	NRS Wheeled Commode Height Adjustable		

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

#### 1.0 Intended Use

The Wheeled Commode range is designed for use as a commode or over a standard toilet. Subject to completion of a suitable risk assessment it can be used for short transfers to move the user over the toilet. It features removable arms to enable side transfers.

#### 2.0 Safety Precautions

Always perform a risk assessment prior to use, including safe moving distance.

Do not exceed the maximum user weight of the product, see table above for details.

Do not use as an alternative to a wheelchair.

Do not use in a shower or as a shower chair.

Do not use the product outdoors. Only use on indoor level surfaces and take care over door thresholds.

Always engage the brakes and swing away or lift off the footrests when the user is transferring onto or off the commode.

Take care to ensure all body parts are inside the perimeter of the commode when passing through doorways.

#### 3.0 Unpacking

Remove packaging and check all parts carefully. If any parts appear damaged do not assemble and contact your supplier immediately. See Table 1 for contents list.

#### 4.0 Assembly Instructions

Please visit

<https://www.nrshealthcare.co.uk/bathroom-aids/toilet-aids/commodos/nrs-wheeled-commode-height-adjustable> for an assembly demonstration.



Figure 1 - Model M66119

Part	L22056	M66119	Qty.
	M99448 N54411		
Main Chassis & Castors	✓		1
Main Chassis		✓	1
Seat Cover	✓	✓	1
Detachable Arms	✓	✓	2
Detachable Back	✓	✓	1
Potty	✓	✓	1
Footrest	✓	✓	2
Adjustable Height Castors		✓	4
Hand Wheel	✓	✓	2
E-Clip		✓	4

Table 1 – Contents List

#### 4.1 M66119 Only

Fit the adjustable height castors to the main chassis by sliding the tube inside the chassis frame, to the desired height and secure using an 'e-clip'.

**Please Note:** the height should be set such that the user feels comfortable but also allows for adequate clearance if being used over a toilet.

Always ensure all four castors are set at equal height.

## 4.2 All Models

Attach the arms by slotting the tube inside the main chassis frame on either side of the seat moulding.

Attach the backrest by slotting the tube inside the main chassis frame and secure in position by inserting and tightening the hand wheels.

*Do not over tighten the hand wheels. Turn until you feel resistance and the screw thread is biting against the leg of the backrest. Do not use tools.*

Attach the footrests to the main chassis frame by fitting over the two studs on either side of the main chassis frame. The footrest will stick out to the side.

Gently push to rotate the footrest to the front of the commode and secure in position by closing the lever shut.

Fit the potty by sliding it onto the metal runners at the rear of the commode and pushing it forwards.

Place the seat pad over the moulded seat.

## 5.0 Footrest Height Adjustment

Detach each footrest from the commode by pulling the lever to allow the footrest to swing out sideways.

Lift the footrest up and off the two studs and turn upside down.

Use the spanner provided to loosen the locking nut on the end of the tube.

Slide the inner tube to the desired height.

Use the spanner to re-tighten the nut and secure the position.

Re-fit the footrest to the commode, see section 4.2.

## 6.0 Cleaning

*NRS Healthcare offer a range of protective gloves and cleaning solutions to assist in the effective decontamination of the potty and commode. Please visit our website or contact Customer Services for more details.*

### 6.1 Commode Potty

Remove the commode potty by carefully sliding it along the metal runners at the rear of the commode.

Place the lid over the potty to prevent spillages and splashing.

Dispose of the contents into a proper sluice or toilet.

**WARNING!** *Never empty the contents into an open drain, sink or basin as this could cause a severe health risk.*

Wash out any residue with a disinfectant solution and rinse thoroughly with fresh water.

Refit the potty to the commode, see section 4.2.

### 6.2 Commode Chair & Frame

Thoroughly wash the commode after each use using a general purpose neutral detergent and warm water.

To disinfect, in particular the seat, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

## 7.0 Maintenance

Inspect the product regularly for signs of wear or damage. Pay particular attention to the upholstery. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Periodically spray a water dispersing agent (e.g. WD40) onto all joints and castors.

Check that all components are correctly fitted and secure. If necessary tighten the nuts, screws and hand wheels.

Check the castors are free to rotate, remove any build-up of dirt/debris.

Frequently check the brakes are operational and there are no signs of wear.

Store the commode on a flat surface, with the brakes locked, out of reach of children.

## 8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months against faulty materials and workmanship. This guarantee does not cover; abuse misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)



**NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:**