



This quick guide is designed to be used in conjunction with the Volker Healthcare Bed Operating and Maintenance Manual.

The Handset Control is a "press" button operation.

To "OPERATE" the bed, press the required function button and hold until the required position of the bed is achieved.

To "STOP" the operation of the bed, release the button.

Handset Functions

Head / Back Section UP

Press to raise the Head / Back Section.

Leg Section UP

Press to raise the Leg Section.

Comfort positioning

Reverse Trendelenburg with entire lying surface tilting head high and feet low. (Chair Position)

Height Adjustment UP

Press to raise the height of lying surface.

Lock the Wheels

Press the height adjustment up button. Wheels will retract. Stop height adjustment a required height.

Head / Back Section DOWN

Press to lower the Head / Back Section.

Leg Section DOWN

Press to lower the Leg Section.

Height Adjustment Down

Press to lower the height of lying surface.

Extend the Wheels

Note: Bed will only go onto its wheels in lowest position.

Press and hold height adjustment "down" button, and the "Wheels" button together to extend the wheels.



Safety Notice.
Do not leave bed on wheels.



Safety Notice.
Bed will only go onto its wheels in lowest position.

Nurse's Control Lockout Box

Height adjustment UP / DOWN

Double-click-function up/down.

By pressing either button twice in quick succession continuous up or down movement of the lying surface can be achieved. Instant stop by pushing any button.

Hand control lock out

Locks all hand control and console unit functions – except Trendelenburg.

Turn knob to lock symbol to lockout handset.

Turn knob to un-lock symbol to allow handset to be operated.

Trendelenburg- Position

Tilts the entire lying surface with the foot section high.

Reverse-Trendelenburg- Position

Tilts the entire lying surface with the head section high.

Lying surface: Head / Back section

Raises and lowers the back section of the lying surface.

Lying surface: Leg section

Raises and lowers the leg section of the lying surface.

Extend the Wheels

Note: Bed will only go onto its wheels in lowest position.

Press and hold height adjustment "down" button, and the "Wheels" button together to extend the wheels.



Safety Notice.
Do not leave bed on wheels.
Bed will only go onto its wheels in lowest position.

Raised Foot Section. Manual adjustment

Pushing Leg section up button moves the bed to the "raised knee"-position. However, Manual setting of the foot section is achieved as follows.

The foot section can be raised manually by lifting foot section to required position. There are 4 (Four) set manual positions which are determined by a ratchet system.

To lower from manual position setting, raise manually foot section to uppermost position and then lower. This will release ratchet positions.




Fault Finding


This flow chart guide is designed to enable the user of the bed to overcome initial operational problems.

Please follow the line diagram to assess the problem or refer to the main operating and maintenance manual.


Section 13 (Trouble Shooting)

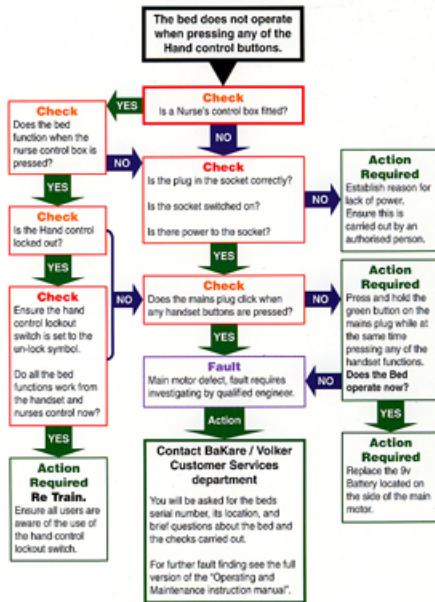

Raising the Assist Side Rail

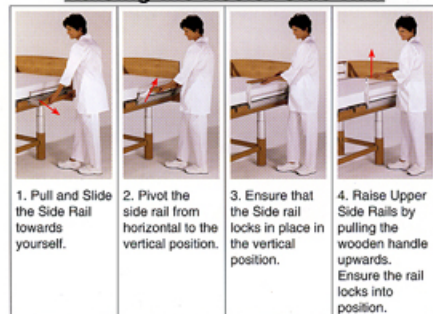


Lowering the Assist Side Rail

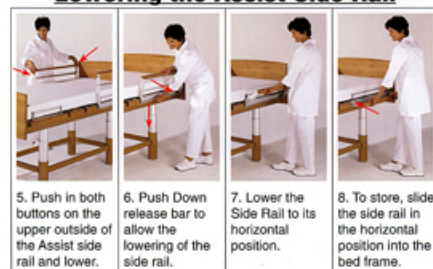



Assist Side Rail Protection

With the Assist Side rails raised, full protection can be achieved by inserting a Plexi-infill.

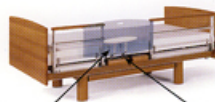

To remove the Plexi - Infill, hold back release pin on base of Infill bar, and pull swinging pin outwards.

This is simply inserted into two location points.

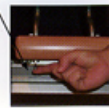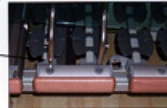

Plexi - Infill Bumper Cover.



The Plexi - Infill can have a bumper cover fitted. This is held in position by Velcro fasteners.

Assist Side Rail Bumper Covers.



The Side Rails can be protected with Slip Over Covers that can be removed as required. These covers are secured in position by Velcro fasteners.

Note: Bumper Covers must be removed before Assist side rails can be lowered and stored.



Before using Side Rails in any configuration, the carer must carry out a true risk assessment of the needs and condition of the patient.

In order to minimise the danger of injury, keep bed in lowest position except when providing care.

Volker Healthcare Ltd / BaKare Beds Ltd
Customer Service Department

Direct Line

01752 512227