# Spa Range



# PUF1000 Urinal Pan - Female Urinal

Long Life Re-usable Plastic Female Urinal Pan for use in Nursing, Rehabilitation, Elderly Care, Community Care and Personal Home Use



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



#### **Product Benefits**

- For female use.
- Lightweight, strong and durable.
- Secure fitting bung to prevent leaks and spills.
- Spout for ease of emptying and cleaning.
- For hand washing only.
- Smooth material surfaces ensure ease of use and ease of cleaning.

## **Product Specification**

- Female urinal pan 1000ml
- Pack size 20
- Pack Weight 5.3kg
- The following standards are applicable:
  - ISO 9001:2008
    - Quality Management System
  - ISO 13485:2003
    - Medical Devices Quality Management System
- Pan is manufactured from Polyethlene (HDPE)
- Bung cap is manufactured from co-polymer non latex

### **Guide to Using Spa Products from Warwick SASCo**

#### **Usage**

- For female usage only.
- For use in a seated position.
- Ensure bung is securely in place before use.
- To empty remove bung.
- Empty after each use.

# Cleaning

- For Hospital Use Wash before first use. Clean in automatic washer disinfectors. Maximum water temperature 100 °C. Do not autoclave bung.
- For further processing information, refer to machine manufacturers instructions and local hospital policy.
- For Personal Use Wash before first use in hot water with detergent. Spa Products are designed for repeated use - guaranteed for 1000 washes. Empty contents immediately after use. Wash in hot water with detergent after each use and allow to dry. Scouring and abrasive cleaners will reduce the performance of the product.

#### Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4  $^{\circ}$ C 25  $^{\circ}$ C.