

Adjustable Platform Bath Step K40



As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement, when using this equipment.

1. Ensure the step is dry and free from soap or oil film.
2. Position step up to side of bath.
3. To adjust the height of the step, remove sections by easing apart each corner using a blunt edge tool. Ensure the remaining sections are firmly pushed together.
4. Stand centrally on the step with both feet.
5. Step directly into the bath.
6. Test for stability.
7. Clean with non-scratch detergent.
8. The product is designed for use in a static mode only.

Maximum operating load



500-kg (70 stone)

 **Langham**
Product

TRENT LANE
CASTLE DONINGTON
DERBY DE74 2NP
ENGLAND
TEL: (01332) 850277
INT: + 44 1332 850277
FAX: (01332) 850366
INT: + 44 1332 850366



K40/0400