



Max User Weight 160kg (25 stone)

Should you have a problem with this device, please quote the batch number attached to the frame.

The product is designed to aid persons with limited mobility / ability or whilst recovering from an injury or illness.

DESCRIPTION

A chrome plated steel frame adjustable in height with underarm support. Four braked castors enable the user to move the frame without fully lifting it and provide a stable platform when locked.

INTENDED PURPOSE

The frame is intended to aid the user in providing underarm support when walking if they are unable to use a traditional walking frame. It is designed so that the user may walk in as upright position as possible. Four braked castors enable the device to be pushed forward during use and can be locked to provide a stable platform. Suitable for indoor use only on flat surfaces.

INSTRUCTIONS FOR USE

When in use, the user should hold the handles and allow the pads to support underarm. The supports are not designed to carry the full user weight. They are intended to provide support during walking. Your OT / healthcare professional will adjust the support to the individuals requirements.

The user should stand within the footprint of the frame and manoeuvre it forward at their own pace and not be pushed. A carer may be required to assist the user depending on the level of patient ability.

The brakes **MUST** be applied when transferring to and from the frame.

SAFETY PRECAUTIONS

The user should have the necessary strength to push the device forward whilst maintaining balance.

Only suitable for use on flat surfaces. Care should be taken when travelling over slopes.

Do not attempt to use on stairs or escalators.

Do not attempt to negotiate kerbs or obstacles without suitable assistance.

Do not carry bags or other objects on the frame.

Before use, ensure all fixings are secure and there is no excessive flex or movement in the frame structure.

Always make sure the frame is level and balances evenly before use.

Make sure the castor wheels rotate freely.

Discontinue use if any of the fixings become loose.

CLEANING

Use a damp cloth or anti-bacterial wipe. Do not use scrapers or abrasive pads. Always clean any residues from hand grips to prevent slippery surfaces.