



AutoSlide is used to aid transfers into and out of car seats. It consists of two sections with surfaces that slide against each other. AutoSlide has a tension strap for fastening at the back. There are handles at the sides for caregivers.

AutoSlide is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

RoMedic's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.

Functional inspection

Visual inspection


Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.

Always read the instructions

Read instructions for all assistive devices used in connection with transfers.
To prevent discomfort, exercise care and caution when using the product.



Functional inspection

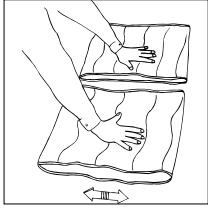

Test AutoSlide's friction by rolling it and noting how easily it glides. If possible, compare with a new AutoSlide.


Test clasp function and handles by applying tension to ensure that the product withstands heavy load.

Placement of AutoSlide



Place AutoSlide with the smaller section's rough surface facing the car seat cushion. Fasten the tension strap around the back support and tighten securely.

Using AutoSlide

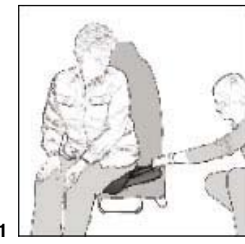

The patient can sit on the seat and then easily lift/turn the legs into the car while gliding all the way back onto the seat (1).

A caregiver can assist from the driver's side of the car by pulling on the handles on the sides of the AutoSlide (from the driver's side) (2).

NOTE! The patient should always wear a seatbelt.

Material:

Polyester: Durable material with low friction.

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

When in doubt, contact your RoMedic representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative, or visit www.romedic.com. Make sure you always have the most recent version of the manual. Manuals are available for downloading from our website; <http://www.romedic.com>

RoMedic AB

Box 640

SE-175 27 Järfälla

SWEDEN

Tel: +46 (0)8-594 773 33

Fax: +46 (0)8-594 773 44

Internet: www.romedic.com

E-mail: office@romedic.com

RoMedic[®]
A company in the Handicare Group