

User Instructions

Bamboo Furniture Raisers

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

To provide a simple solution for raising the height of household furniture. Suitable for furniture with 4 legs or castors.

2.0 Safety Precautions

Take care when fitting the raisers, ask for assistance if furniture is heavy or bulky.

Do not exceed the safe working load for the set of 4 raisers, 60st (380kg).

Do not attempt to move raised furniture which is occupied.

3.0 Unpacking

Remove packaging and check all products carefully. If any items appear damaged do not use and contact your supplier.

4.0 Fitting

Lift the corner of the furniture to be raised and slide the raiser underneath.

Locate the furniture leg or castor in the recess in the top of the raiser.

Please Note: P11644 offers two recess diameters, choose the diameter most suitable for the furniture.

Repeat for all four legs/castors and check the furniture is stable before use.

5.0 Cleaning

Clean regularly using a damp cloth to remove the build-up of dust and dirt.

P11644 – 4inch cube

P11656 – 6inch with base



6.0 Maintenance

Check the raisers for signs of wear. Any suspected damage should immediately be reported to the supplier and the product taken out of use.

7.0 Product Disposal

Product is manufactured from FSC approved Bamboo and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare, Sherwood House, Cartwright Way, Forest Business park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk