

**Information Sheet. Product Code: 3308**

**Product: Chemical Commode  
Chair**



**User weight limit  
160kg (25 stones)**

Should you have a problem with this device, please quote serial number attached to the frame.

**DESCRIPTION:**

A sturdy metal commode with a hinged black plastic toilet seat over a removable 20-litre bucket with lid.

The legs can be adjusted in height from 20 ½" to 26 ½" by gently prising out the pin clips on each leg and sliding the telescopic extension to the desired height.

**INTENDED PURPOSE:**

To be used as a portable commode with an easily disposable toilet facility.

**INSTRUCTIONS FOR USE:**

**Sitting-**

Approach the commode pan, turn around and feel the horizontal bar with the back of your legs.

Put both hands on the armrests and lower oneself gently on the commode seat.

**Standing-**

Put both hands on the armrests and push up into standing position.

**FITTING INSTRUCTIONS:**

Place in a convenient room and position for client's use.

Adjust the surround until the armrests are at a comfortable height. Do this by gently prising out the pin clips on each leg and sliding the telescopic extension to the desired height.

**SAFETY WARNING:**

Ensure the legs have been adjusted to the same height by counting the number of holes in view.

Never stand on the commode.