



Max. User Weight 190kg (30 st)

Should you have a problem with this device, please quote the batch number attached to the frame.

*Description:* A stool type seat in moulded plastic which has large rubber suckers to give a firm grip. Suitable for use in plastic baths.

Available in two heights:-  
15cm (6")  
20cm (8")

*Intended Purpose:*

To provide a secure platform to aid the user in everyday bathing.

*Instructions for Use:* A Derby bath seat may be used in conjunction with a bath board.

1. Step into your bath using any available rail for assistance.
2. Using your arms as well as your legs for support, lower yourself down onto the bath seat.
3. Sit on the seat to wash.
4. When you have finished washing, let the water out of the bath before getting out.

*Fitting Instructions:* Place the seat in the bath ensuring the four rubber suckers hold securely. There should be no movement of the bath seat.

*Safety Warning:* We advise that oils and bath creams should not be used.  
Do not stand on the seat.

*Cleaning:* Use a damp cloth or anti-bacterial wipe. Do not use scrapers or abrasive pads. Always clean any residues to prevent slippery surfaces.