# YOU-Q QLASS

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© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for possible errors in the information or the consequences thereof. Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

2016-01

<b>1</b>	<b>Forward</b> .....	<b>23</b>
<b>2</b>	<b>General description</b> .....	<b>23</b>
	2.1 Product description QLASS .....	23
<b>3</b>	<b>General operations</b> .....	<b>24</b>
	3.1 Seat cushion: Removing / replacing .....	24
	3.2 Backrest: Removing / replacing .....	24
	3.3 Legrest: Removing / replacing .....	25
	3.4 Armrest: Removing / replacing .....	25
	3.5 Backrest (frame): Folding down .....	25
	3.6 Headrest .....	26
<b>4</b>	<b>Setting the seating system</b> .....	<b>27</b>
	4.1 Seat depth setting .....	27
	4.2 Seat width setting.....	28
	4.3 Legrest settings.....	29
	4.4 Armrest settings .....	31
	4.5 Headrest .....	33
	4.6 Hip belt .....	34
<b>5</b>	<b>Adjustments</b> .....	<b>35</b>
	5.1 Comfort legrests.....	35
	5.2 Headrest with quick release levers .....	35
	5.3 Electrical adjustments .....	35
<b>6</b>	<b>Getting in and out of the wheelchair (transfer)</b> .....	<b>36</b>
<b>7</b>	<b>Reducing the wheelchair size</b> .....	<b>37</b>
<b>8</b>	<b>Maintenance</b> .....	<b>37</b>
	8.1 Maintenance table.....	37
	8.2 Cleaning the upholstery .....	37

# 1 Foreword

This user manual

The user manual for this electric wheelchair consists of three booklets. Read the entire user manual carefully before taking the product into use. The information provided by this manual is essential for the safe use and proper care (cleaning) of the wheelchair.

The general user manual (this booklet)  
The user manual for the seating system (QLASS)

The user manual for the controller

If one of these user manuals was not included with your wheelchair, please contact your dealer immediately. In addition to this user manual, there is also a service manual for qualified specialists.

This general user manual will refer, where necessary, to one of the other user manuals. This is indicated as follows:

 **GENERAL**

 **CONTROLLER:** Refers to the user manual for the controllers.

**Strictly follow the procedures with a caution symbol! These procedures can if they are not executed properly - result in human injury or damage to the product or the environment.**

## For further information

Please contact your local, authorised Sunrise Medical dealer if you have any questions regarding the use, maintenance or safety of your wheelchair. In case there is no authorised dealer in your area or you have any questions, contact Sunrise Medical either in writing or by telephone.

For information about product safety notices and product recalls, go to [www.SunriseMedical.eu](http://www.SunriseMedical.eu)

## CE declaration of conformity

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

## Warranty

The warranty stipulations applicable to the seating system are identical to and incorporated in the wheelchair manual.

# 2 General description

## 2.1 Product description QLASS

The wheelchair is equipped with the QLASS seating system. The QLASS seating system provides excellent support for the head, torso, arms and legs. Optimal body support ensures maximum independence for the wheelchair user.

The wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for the sitting posture.

**Please study this manual carefully before you use the wheelchair.**

### 3 General operations

#### 3.1 Seat cushion: Removing / replacing

**Removing the seat cushion** (fig 3.1).  
Pull the seat cushion upward to remove it from the seat.

**Replacing the seat cushion** (fig 3.2).  
Click the seat cushion onto the seat frame.

**Check if the seat is clicked in place securely.**

#### 3.2 Backrest: Removing / replacing

**Removing the backrest** (fig 3.3).  
Push the Q-clips up.  
Take away the backrest.

**Replacing the Backrest** (fig 3.3).  
Position the backrest to the frame.  
Push the Q-clips back in place.

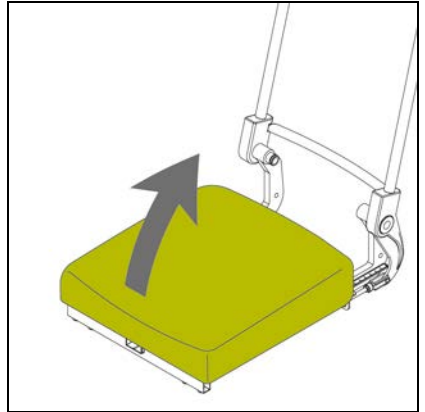
Figure 3.1

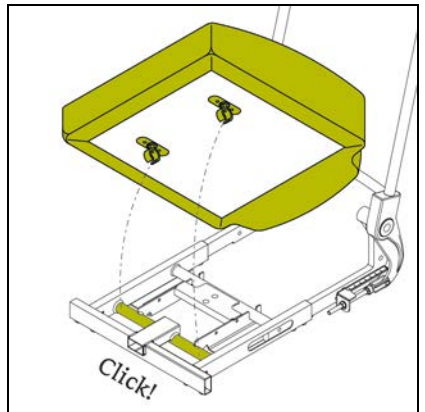
Figure 3.2

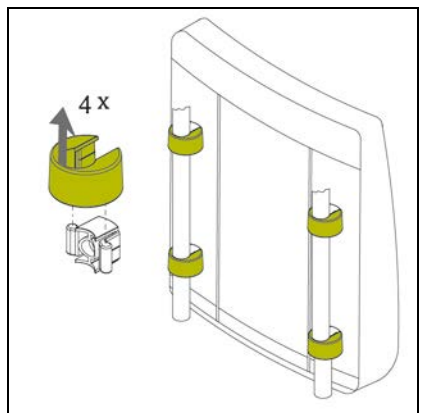
Figure 3.3

### 3.3 Legrest: Removing / replacing

**Removing** (fig 3.4).

1. Loosen the calf strap.
2. Push the catch to the rear in order to unlock the leg rest.
3. Swing the leg rest outwards.
4. Remove the leg rest from its bracket by lifting.

**You should hear a click when replacing the leg rest. This indicates that the leg rest has locked into place.**

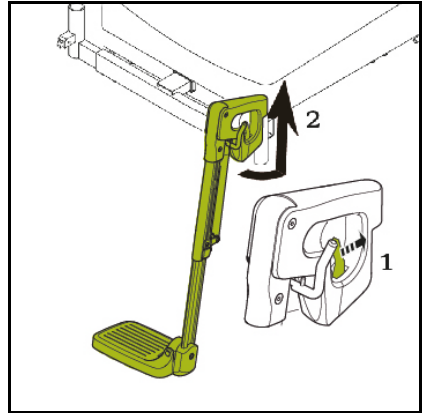
Figure 3.4

### 3.4 Armrest: Removing / replacing

**Removing the armrest** (fig 3.5).

1. Push the lever to the back to unlock the armrest.
2. Take the armrest out.

**Replacing the armrest** (fig 3.3).

1. Place the armrest into the armrest holder.
2. Lock the armrest by putting the lever in a vertical position.

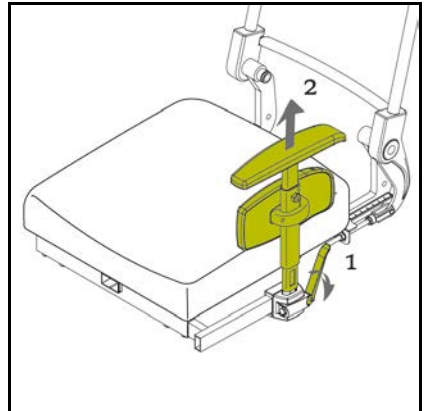
Figure 3.5

### 3.5 Backrest (frame): Folding down

1. Release the circlip from the adjustment bar (fig 3.6).
2. Pull the lock pin out of the hinge joints (fig 3.6).
3. Fold down the backrest (fig 3.6).

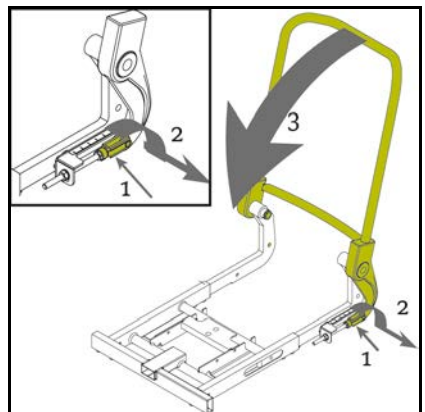
Figure 3.6

### 3.6 Headrest

#### Removing the headrest (fig 3.7).

1. Push the lever up to unlock the headrest.
2. Take the headrest out.

#### Replacing the headrest (fig 3.7).

1. Place the headrest into the head rest holder.
2. Lock the headrest by putting the lever in a vertical position.

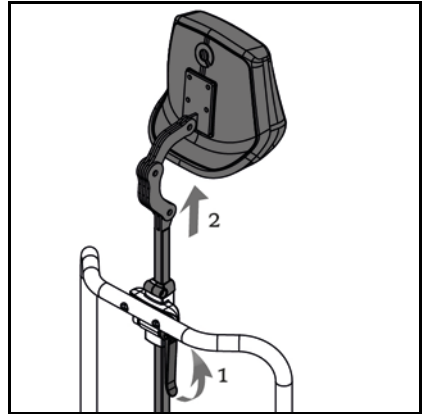
Figure 3.7

## 4 Setting the seating system

### 4.1 Seat depth setting

#### Seat depth by backrest frame (fig 4.1).

1. Loosen the Allen bolts on both sides of the seat frame a half turn (180°).
2. Loosen the Allen bolts on both sides of the seat frame a half turn (180°).
3. Tighten the Allen bolts.

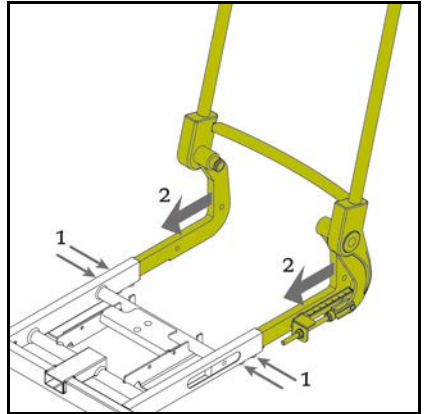
Figure 4.1

#### Seat depth by positioning seat cushion (fig 4.2).

1. Remove the seat (fig 3.1).
2. Place the clamps in the right position.
3. Click the seat cushion onto the seat frame.

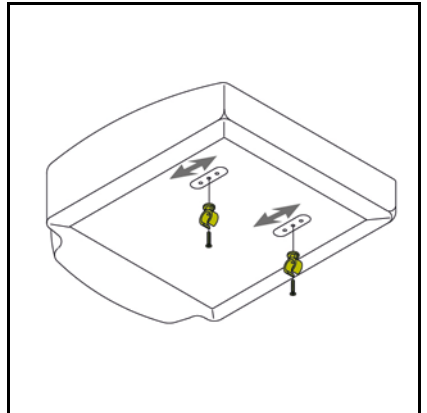
Figure 4.2

**Check if the seat is clicked in place securely.**

**Make sure that there is at least 2 cm of free space between the back of the knee and the front of the seat to ensure good blood circulation.**

## 4.2 Seat width setting

### Seat width setting by armrests (fig 4.3).

1. Loosen the bolts.
2. Slide the armrest in the desired position.
3. Tighten the bolts.

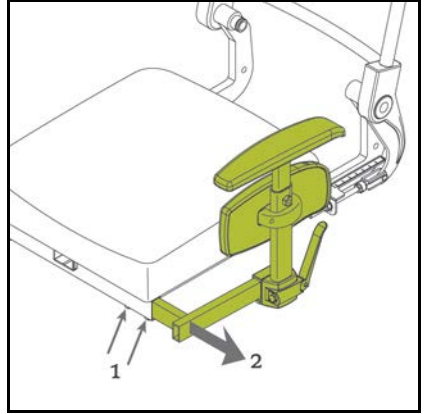
Figure 4.3

### Seat width setting by legrests (fig 4.4).

1. Loosen the bolts.
2. Slide the legrests in the desired position.
3. Tighten the bolts.

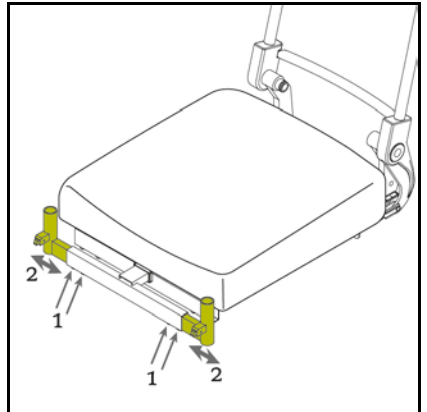
Figure 4.4

### Backrest angle setting

See figure 4.5

1. Loosen the bolts on the left side of the seat frame.
2. Set the desired angle of the backrest.
3. Tighten the bolts.

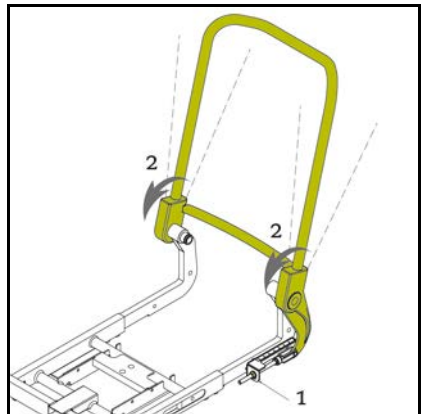
Figure 4.5

### 4.3 Legrest settings

#### Lower leg length (fig 4.6).

1. Loosen the bolt.
2. Slide the footplate in the desired position.
3. Tighten the bolt.

**Make sure that there is sufficient free space under the footplates to operate the wheelchair over obstacles.**

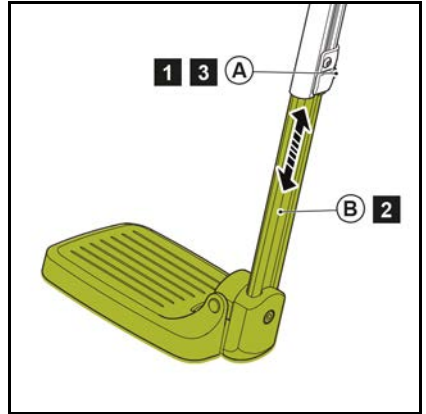
Figure 4.6

#### Angle of the footplate (fig 4.7)

1. Loosen the bolt.
2. Set the footplate in the desired angle.
3. Tighten the bolt.

#### Legrests width setting (fig 4.4)

See chapter 4.2

#### Seat depth by legrests (fig 4.8).

1. Loosen the bolt.
2. Slide the legrest holder in the desired position.
3. Tighten the bolt.

- **Make sure that the castor wheels do not touch the footplates when manoeuvring. If this is the case, the legrests must be brought forward. There must be at least 1 cm between the castor wheels and the footplates.**
- **Make sure the legrest bracket does not push into the back of the knees causing discomfort. If this is the case, the seat must be adjusted forward.**

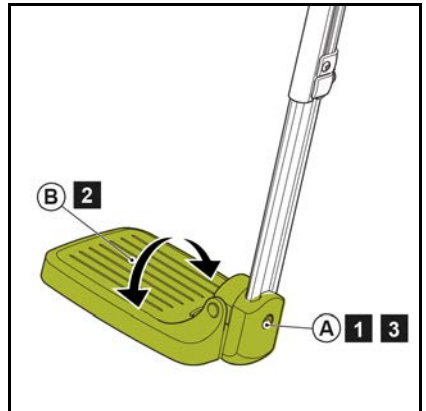
Figure 4.7

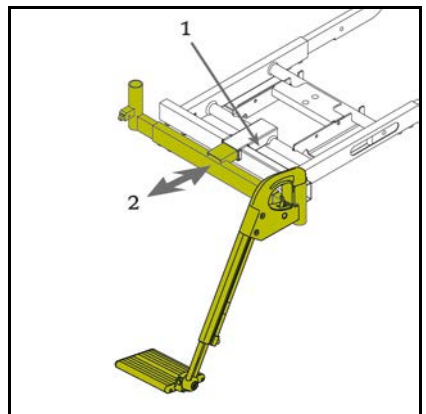
Figure 4.8

**Calplate height, width and depth setting (fig 4.9).**

1. Loosen the bolts.
2. Slide the calplate in the desired position.
3. Tighten the bolts.

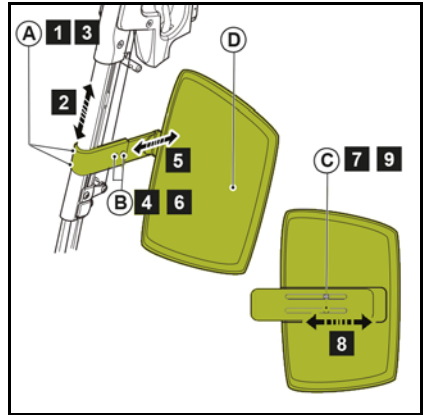
Figure 4.9

## 4.4 Armrest settings

### Armrest height setting (fig 4.10).

1. Loosen the bolt.
2. Slide the armrest in the desired position.
3. Tighten the bolt.

**Make sure that the arms are supported well, without the shoulders being pushed upward.**

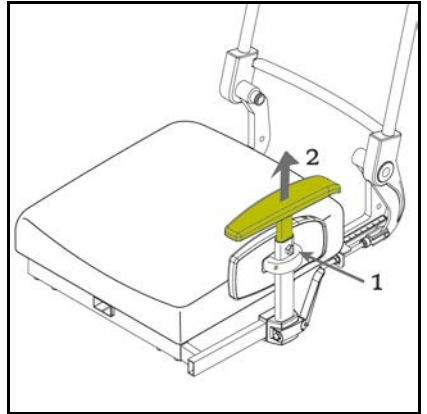
Figure 4.10

### Armrest depth setting (fig 4.11).

1. Loosen the bolts.
2. Slide the armrest in the desired position.
3. Tighten the bolts.

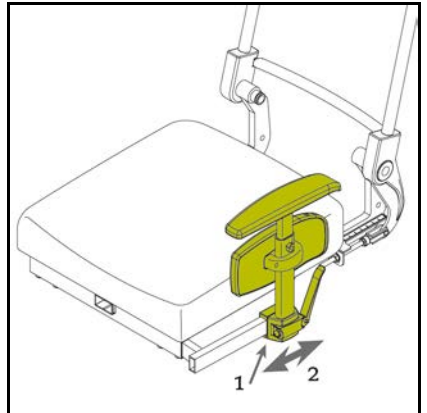
Figure 4.11

### Sideplate height setting (fig 4.12).

1. Loosen the bolts.
2. Slide the armrest in the desired position.
3. Tighten the bolts.

### Armrest width setting (fig 4.3)

See chapter 4.2

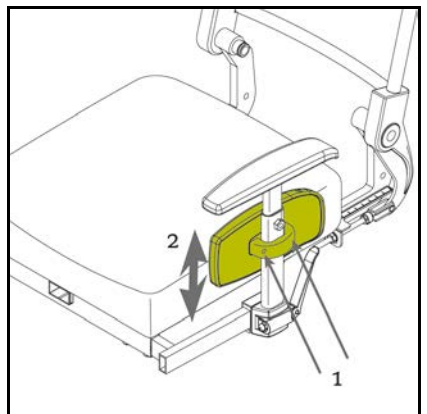
Figure 4.12

**Controller depth setting (fig 4.13).**

1. Loosen the bolt.
2. Slide the control in the desired position.
3. Tighten the bolt.

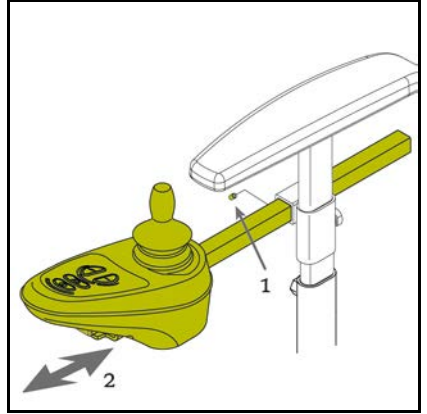
Figure 4.13

**Controller height setting (fig 4.14).**

1. Loosen the bolt.
2. Slide the control in the desired position.
3. Tighten the bolt.

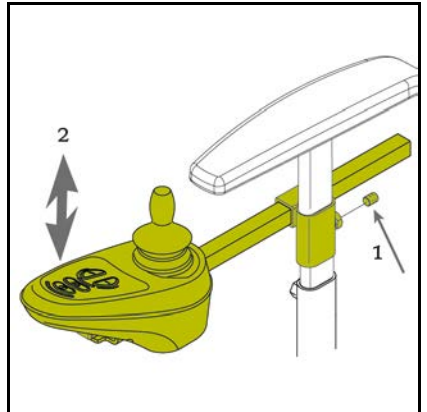
Figure 4.14

## 4.5 Headrest

### Height (fig 4.15).

1. Push the lever up to unlock the headrest.
2. Slide the headrest in the desired position.
3. Lock the headrest by putting the lever in a vertical position.

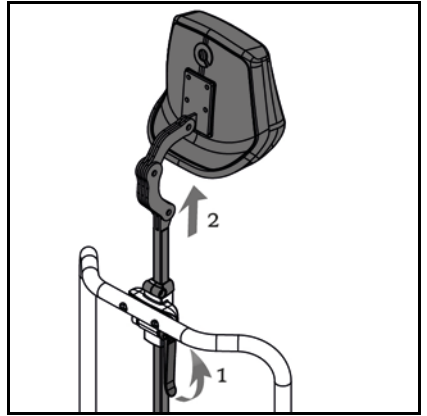
Figure 4.15

### Memory block (fig 4.16).

1. Loosen the bolt.
2. Slide the memory block in the desired position.
3. Tighten the bolt.

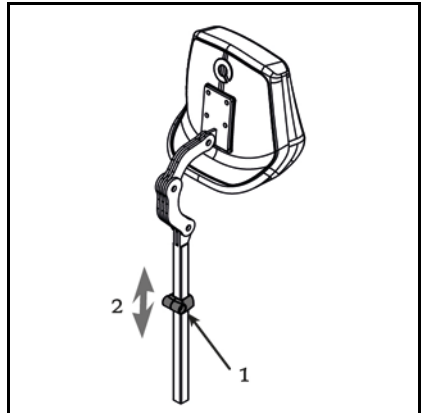
Figure 4.16

### Position (fig 4.17).

1. Loosen the bolts (x3).
2. Put the headrest in the desired position.
3. Tighten the bolts (x3).

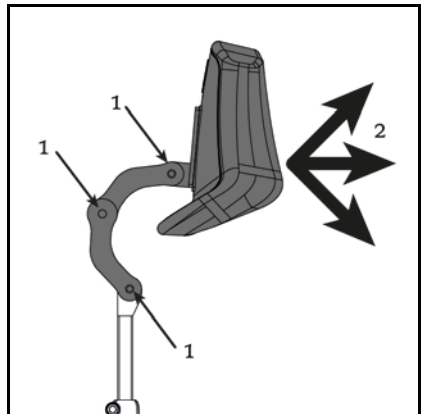
Figure 4.17

## 4.6 Hip belt

It is possible for the wheelchair to be fitted with a positioning belt. The hip belt offers extra seating stability under normal use.

The hip belt is attached as follows (figure 4.18):

1. Attach the belt buckle using a nut and bolt.
2. Pass the belt through the buckle.

**Check that the hip belt is correctly adapted to the wheelchair user. The belt should lie firmly over the hip at an angle of approx. 45° (fig 4.19). Placing the belt over the hip, against the soft skin of the stomach, can result in the wrong sitting position and the user sliding forward in the wheelchair (figure 4.20).**

**Make sure the belt is clean and that the locking mechanism works at all times. The belt and locking mechanism can be cleaned using a damp cloth.**

**Never use the hip belt as a safety belt when transporting the user in the wheelchair.**

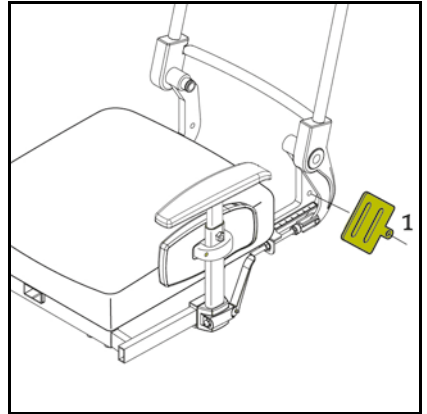
Figure 4.18

Figure 4.19: Right position hip belt

Figure 4.20: Wrong position hip belt

## 5 Adjustments

### 5.1 Comfort legrests

Knee angle (fig 5.1).

The angle of the comfort leg rests can be adjusted independently of each other.

Push handle forwards.

Lift your leg.

Release the handle when in the desired position.

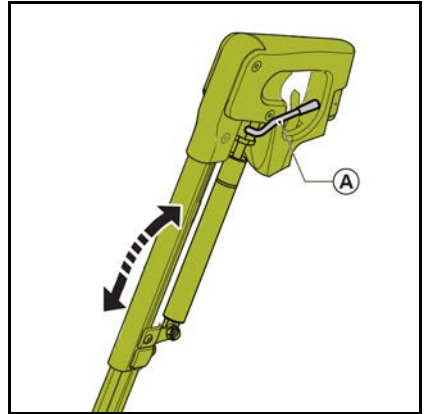
Figure 5.1

### 5.2 Headrest with quick release levers

Position (fig 5.2).

Loosen the clamps (x3) by a few turns.

Adjust to the desired position.

Tighten the clamps (x3) securely.

Figure 5.2

### 5.3 Electrical adjustments

If applicable:

Powered tilt in space

 CONTROLLER

Powered seatlift

 CONTROLLER

Powered backrest recline

 CONTROLLER

Powered legrests

 CONTROLLER

Figure 5.3

## 6 Getting in and out of the wheelchair (transfer)

**Do not stand on the foot-plates. These have not been designed to accommodate the full weight of a person. In addition, this could cause the wheelchair to topple over.**

### Forward transfer

1. Flip the footplates up (fig 6.1).
2. Swing the calf plates to the side (if applicable) (fig 6.1).
3. If possible, swing the legrests to the side (fig 6.2).
4. Seat yourself in the chair.

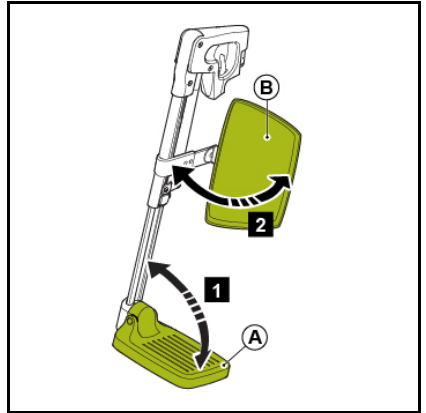
Figure 6.1

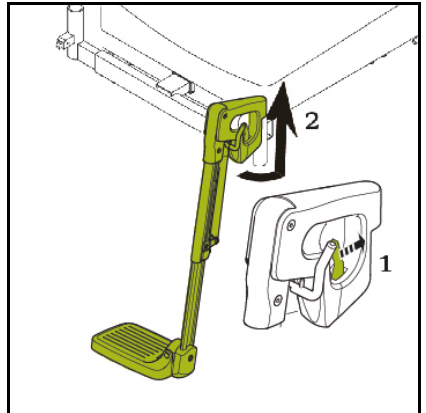
Figure 6.2

### Sideways transfer

1. Push the lever to the back to unlock the armrest (fig 6.3).
2. Take the armrest out (fig 6.3).
3. Seat yourself in the chair.

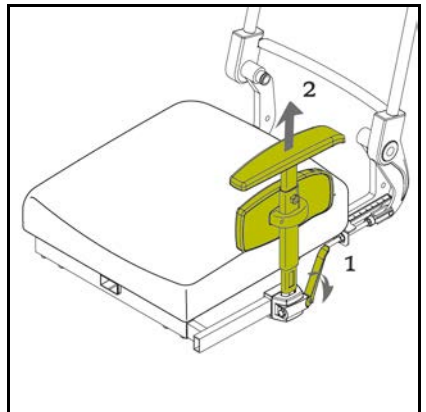
Figure 6.3

## 7 Reducing the wheelchair size

The wheelchair can be reduced in size for transportation without a passenger. Any components of the wheelchair that can easily be detached should be removed. Store such components in a safe place. The backrest can also be folded down if the car's boot space is small.

1. Remove the seat cushion. See ch. 3.
2. Remove the backrest. See ch. 3.
3. Remove the legrest. See ch. 3.
4. Remove the armrests. See ch. 3.
5. Remove the headrests. See ch. 3.
6. Flip the backrest frame down. See ch. 3.

## 8 Maintenance

### 8.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Monthly	Cleaning the upholstery, if necessary. See 8.2.

### 8.2 Cleaning the upholstery

Clean the upholstery with a damp cloth and household soap. Then wipe the upholstery dry with a soft dry cloth.

**Do not use organic solvents such a thinner, benzene or white spirit.  
Do not dry clean, iron or spin-dry the upholstery**

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Sunrise Medical HCM B.V.