

User Instructions for Shower Stools

S10 – S11 – S14

Introduction

Congratulations on your purchase of the Cefndy Healthcare Shower stool. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

The shower stool range is designed to aid users whilst showering. The stools are height adjustable at 25mm intervals and the feet are non-slip and non-marking for extra safety. All stools have a clip on/clip off seat for easy cleaning. The S14 model comes with a detachable back and the S11 model has a gap fronted seat.

All structural joints are fully welded. This feature is to aid product hygiene by reducing the possibility of bacterial entrapment as well as ensuring structural integrity. The steel frame work is also coated in an anti-bacterial powder coated finish.

Specifications

| | S10 | S11 | S14 |
|-------------------------------|--------------|--------------|--------------|
| Seat Height | 450 – 600mm | 490 – 640mm | 450 – 600mm |
| Seat width & depth | 380 x 280mm | 440 x 380mm | 380 x 280mm |
| Width between the arms | 500mm | 562mm | 500mm |
| Footprint | 525 x 560mm | 380 x 560mm | 525 x 560mm |
| Maximum user weight | 159kg / 25st | 115kg / 18st | 159kg / 25st |



S10



S11



S14

Pack Contents

- Shower Stool
- User Instructions

Assembly Instructions

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Correct Use

The back may have been located in an inverted position for transit. If so remove the two safety pin clips located on the back between the hanging brackets, slightly unscrew the hand wheels as above, fully withdraw the back from the brackets. Then rotate the back to an upright position and slide the back into all four brackets, insert the safety pin clips & repeat the adjustment process to locate and secure the back in the correct position.

To adjust the Shower Stool height

- Remove the pin clip from each leg individually
- Select the height by positioning each extension leg individually, aligning the holes in the extension leg and frame
- When the required height is selected re-fit the pin clip into the appropriate hole ensuring the pin is fully engaged
- Repeat the above on the other three legs
- Ensuring all four pin clips are fully engaged and check that all four legs are set to the same height

To adjust the removable back (S14 only)

- Unscrew the handle wheels under the seat (clockwise) around half one full turn
- The detachable back should now slide freely in the locating brackets
- Once the desired position is reached re-tighten (anti-clockwise) the four hand-wheels to lock the back in position
- As an added safety feature pin clips are inserted to the front of the forward hanging brackets to stop the back from sliding out of the brackets **ensure these pin clips are always inserted when the product is in use**

Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.

Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that rails are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the inside of the rail as it could cause permanent rusting of the rail (leave rails in a warm environment for several hours to ensure all moisture evaporates).
- We cannot offer advice on industrial washing machine with regards to specific settings as these will be dependent on the machines manufacturer's instructions.

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- This product is intended for users up to a maximum of 159kg/25st (S10 & S14) or 115kg/18st (S11) in weight.
- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and all other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of in line with local recycling policies.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 01824 706005

E-mail: cefndy.sales@denbighshire.gov.uk