



**Instructions for the safe use and fitting of
 SmitCare Threshold Ramps**

Suitable for Wheelchairs and Scooters

**Code: TR2 (2 foot Threshold Ramp)
 TR3 (3 foot Threshold Ramp)**

Manufactured in China for SmitCare

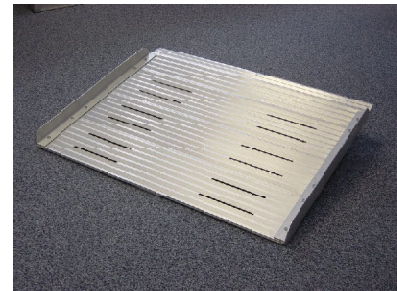


Fig. 1

Important Notes

A risk assessment should be carried out as to the suitability of this product particularly with new users. The following risks should be considered: Entrapment of fingers. Falling/Tripping. Has the product been prepared for safe usage? Will the user be safe on the ramps with their equipment?

Dimensions

Threshold Ramps are available in the following lengths:

Length	Width	Weight	Safe Weight Bearing
2 ft	29 ³ / ₄ " (755 mm)	4.8 kg	350 kg
3 ft	29 ³ / ₄ " (755 mm)	7.1 kg	350 kg

User / Fitting Instructions

These ramps should not be used for gradients of more than 1 in 12 for a permanent ramping solution (Fig.1).

To position the ramp, place the non-slip rubber tread (on the underside of the top flange) onto the step and lower the other end of the ramp to rest on the ground.

Important - Check that the rubber tread is as far onto the step as possible.

Benefits

Suitable for most exterior ramping requirements.

Anti-ice drainage slots.

Important Notes

These ramps are supplied with pins for a permanent or semi-permanent fixture if required. Should this method of fitting be necessary make sure that they are fitted by a qualified handyman/tradesman or Occupational Therapy Technician.

Cleaning

Keep your threshold ramps in good order. Clean regularly with a soapy solution and dry with a cloth.

Questions / comments? Please contact our Customer Services
 department on Lo-call : **0845 1303 747**