

Supaseat

RANGE

From walking stick to restful seat in an instant!

Thank you for choosing a Linden Supaseat. We wish you many years of happy use. For your complete safety please read all instructions:

- Supaseat – Observer, Explorer, Foldaway & Gripseat
Walking height 850mm (33¹/₂") to 1030mm (40¹/₂")
Seat height 650mm (25¹/₂") to 830mm (32¹/₂")
- Maximum user weight: 125kg

Your Supaseat is intended for use only as a one person seat.
Do not use it for anything else.


CAUTION: Single tube seatsticks are for those who can balance safely on the seat and form a stable tripod with their weight shared evenly between seat & feet. If you have a disability or condition which may affect your balance, always seek medical advice before use.



- 1 . Height is adjusted by pressing in both spring buttons and sliding the foot to the desired length. Ensure both buttons are locked in a set of holes, before using.
- 2 . Sitting on the seat:



Place the seat behind you with the base approximately 45cm (18") away. Lean the seat towards you and sit slowly with your feet approximately 45cm (18") apart.

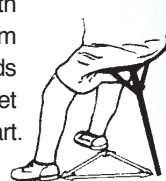

FOLDAWAY (pictured below)

Firstly remove the point & plate foot from the bag. Then remove seat with ferrule foot making sure to hold on to both seat & tubes. Ease tubes out so that they slot together.



GRIPSEAT

The Gripseat with its magnetic end is ideal for collecting used Shotgun cartridges.

MODELS AVAILABLE

Observer
(rubber ferrule)

Explorer
(point & plate for soft ground only)

Foldaway
(rubber ferrule & point & plate)

Gripseat
(point & plate for soft ground only, magnetic end)

COLOURS AVAILABLE
Black, Brown or Green

