

Information Sheet

Product Code: 3325,3326 and 3327
**Product: Elsan Toilet frame, Elsan Bristol Toilet,
Elsan Blue Fluid.**



**Maximum user weight
160kgs (25 stones)**

Should you have a problem with this device, please quote the batch number attached to the frame.

DESCRIPTION

The frame is manufactured from plastic coated steel tube with plastic armrest and adjustable height legs. The Elsan Bristol Toilet is placed on the steel base plate of the frame.

INTENDED PURPOSE

Assisting in the toileting of persons where normal toilet facilities are not available or where there is a need to have a flexible alternative from normal toilet facilities.

INSTRUCTIONS FOR USE:

Place the Elsan Bristol Toilet on the steel base of the frame. Ensure the surface where the frame is being placed is even. Adjust the legs by removing the E clip from each leg in turn and check the number of holes on each leg for length replace the E clip or set the height required.

60 ml of Elsan fluid should be added to the waste holding tank for every 10litres of waste tank capacity. Then add 1-3 litres of water. Always ensure that solid matter is covered, adding more water if necessary. More Elsan blue may be required dependent on the frequency of use and temperature. Never add Elsan Blue to the fresh water tank. Empty the waste at a designated point, and then rinse with fresh water.

Do not over fill the waste bucket so that it becomes too heavy to lift.

SAFETY PRECAUTIONS:

In case of contact with eyes rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing and gloves. If swallowed seek medical advice immediately and show the container or the label.

Never stand on the product.