

Information Sheet. Product Code: 4212 Product: Swedish Bath Rail



User Weight Limit: 125kg (20 st)

Should you have a problem with this device, please quote the batch number attached to the frame.

DESCRIPTION:

This bath rail is fixed to the side of the bath and is adjustable in height.

Height adjustable from 25" - 30"

Width of handle 9"

INTENDED PURPOSE:

This bath-side rail provides a firm handle just above the side of the bath, running parallel to the side, primarily to assist getting in and out of the bath from standing. The rail can be used in conjunction with a bath seat.

INSTRUCTIONS FOR USE:

Adjust the legs to the required height. Adjust the two arms to fit snugly over the rim of the bath and lock into position with the wing nuts. The unit MUST be screwed firmly to the floor. Keep clean with a detergent suitable for bathroom fittings.

FITTING INSTRUCTIONS:

The rail screws to the floor and clamps over the side of the bath. The equipment needs to be fixed to floor by a contractor.

<u>SAFETY WARNING:</u>

These rails are not designed to be used to pull up from sitting in the bottom of the bath.

ROMA MEDICAL AIDS LTD. York road, Bridgend Industrial Estate, Bridgend, CF31 3TB. TEL: 01656 674488, FAX: 01656 674499.