



Max: 135 kg /297 lbs

EasyGlide is used for transfers in the seated position between wheelchair and bed, toilet or car seat. EasyGlide is available in a range of sizes to meet most transfer needs. The smaller model is often used by active patients who can manage transfers on their own.

EasyGlide is part of a series of different models of transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

RoMedic's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.



## Functional inspection

### Visual inspection

Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.




## Always read the instructions

Read instructions for all assistive devices used in connection with transfers.

To prevent discomfort, exercise care and caution when using the product.




## Functional inspection

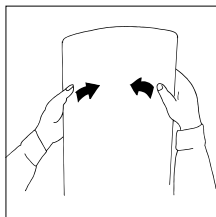


Carefully bend the EasyGlide to check strength and durability. The mat should be both elastic and rigid. Discard the EasyGlide when its strength and durability are diminished.

## Max. weight

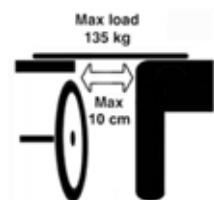

60x33, 60x25, 45x20

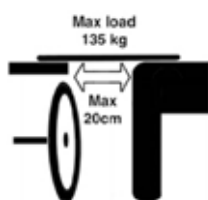

75x33

A maximum patient weight of 135 kg applies where the distance between transfer locations is 10 cm or 20 cm, depending on product size. 10 cm applies for EasyGlide 60x33, 60x25 and 45x20. 20 cm applies for EasyGlide 75x33.

## Placement/removal of EasyGlide



Place the wheelchair at an angle towards the side of the bed, and as close to the bed as possible. Remove the armrest that is closest to the bed and remove leg supports. Place the EasyGlide like a bridge between the bed and the wheelchair. If it is an EasyGlide model on which one long side can be bent, this should be placed against the wheelchair wheel. To position the EasyGlide under the patient, lean the patient slightly in the opposite direction.



When the transfer is complete, EasyGlide is removed by bending upwards (see image).

## Using EasyGlide



Active patients can transfer themselves with the help of EasyGlide. Grasp the armrest and lean over towards the wheelchair. It is important to remember to lean once more towards the bed when the transfer is complete, so as to be able to move back in the middle of the chair and slide all the way back.



If the patient requires support and assistance during the transfer, the EasyGlide should be complemented with an EasyBelt/FlexiBelt.



The caregiver can work from behind or in front of the patient.



EasyGlide can also be used for transfers into and out of a car.

**Material:**

Polyethen

**Care of the product:**

Clean EasyGlide with water and disinfectant.

**Size:**

5020	60 x 33
5050	75 x 33
5060	60 x 25
5070	45 x 20

When in doubt, contact your RoMedic representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative, or visit [www.romedic.com](http://www.romedic.com). Make sure you always have the most recent version of the manual. Manuals are available for downloading from our website; <http://www.romedic.com>

**RoMedic AB**

**Box 640**

**SE-175 27 Järfälla**

**SWEDEN**

**Tel: +46 (0)8-594 773 33**

**Fax: +46 (0)8-594 773 44**

**Internet: [www.romedic.com](http://www.romedic.com)**

**E-mail: [office@romedic.com](mailto:office@romedic.com)**

**RoMedic®**  
A company in the Handicare Group