

# Quattro

SEAT RANGE

## From walking stick to restful seat in an instant!

Thank you for choosing a Linden Quattro seat. We wish you many years of happy use. For your complete safety please read all instructions:

- Quattro 32 – Walking height 815mm (32")  
Seat height 480mm (19")
- Quattro 34 – Walking height 865mm (34")  
Seat Height 525mm (20<sup>3</sup>/<sub>4</sub>")
- Quattro 37 – Walking Height 940mm (37")  
Seat Height 590mm (23<sup>1</sup>/<sub>4</sub>")
- Maximum user weight: 125kg

Your Quattro seat is complete with four rubber ferrules and has been designed for use on level, non slippery surfaces.

N.B. Regularly check that all rubber ferrules are in good condition. For your safety NEVER use the seat with any ferrule missing.



1. To use as a seat: Open fully and ensure all feet are in contact with the ground.
2. Sit down slowly. For maximum stability it is important when sitting to have your legs in front of you with both feet firmly on the ground.
3. The Quattro seat is intended for use only as a one person seat. Do not use it for anything else.



## SPORT SEAT MODELS

The Sport seat models with the additional point and plate feet can be used on soft ground by removing all rubber ferrules. Replace all ferrules for use on hard ground.



## MODELS AVAILABLE

Polished aluminium frame and coloured canvas

or

‘Deluxe’  
painted frame to match seat colour –

COLOURS AVAILABLE  
Black or Green

