

## User Instructions for Cefndy Trolleys

M101 – M103 – M166

### Introduction

Congratulations on your purchase of the Cefndy Healthcare Trolley. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

The trolleys are all height adjustable and are suitable for different needs. They are intended to be easy to use, manoeuvrable and easy to clean. The large pulpit style handles on the trolleys give users a sense of controlled movement, and the trays are all removable for easy cleaning. The M101 & M166 plastic trays have retaining edges and the M101 features a cup recess.

### Specifications

	<b>M101</b>	<b>M103</b>	<b>M166</b>
<b>Height Range</b>	880 – 1030mm	880 – 1030mm	880 – 1030mm
<b>Top Tray Height</b>	665 – 815mm	635 – 785mm	675 – 820mm
<b>Bottom Tray Height</b>	390 – 540mm	365 – 515mm	400 – 550mm
<b>Depth of Frame</b>	610mm	610mm	610mm
<b>Width of Frame</b>	440mm	440mm	440mm
<b>Tray Size</b>	460 x 340mm	395 x 355mm	460 x 340mm
<b>Weight Limit</b>	159kg / 25st	159kg / 25st	159kg / 25st



### Pack Contents

- Trolley
- 2 x Trays
- User Instructions

### Assembly Instructions

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

### Correct Use

#### Castor Check:

- The castors should be occasionally checked to ensure correct operation. For all four castors please check the fitting in the frame tube to ensure they are a secure fit.
- Check their rotation and free swivel.

#### Height Adjustment:

- Remove the pin clips holding the legs.
- Select the desired height for each leg and align the selected hole with the extension leg to ensure the pin clip goes fully through.
- Replace pin clips making sure they are fully located. Ensure all legs are set to the same height.
- The trolley is now ready for use.

### Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.

### Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that rails are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the inside of the rail as it could cause permanent rusting of the rail (leave rails in a warm environment for several hours to ensure all moisture evaporates).
- We cannot offer advice on industrial washing machine with regards to specific settings as these will be dependent on the machines manufacturer's instructions.

### Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

### Warning

- **Trolleys must NOT be used as walking aids.**
- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and all other relevant considerations.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

### Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

### Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non-hazardous and can be disposed of in line with local recycling policies.

### Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare  
Cefndy Road  
Rhyl  
Denbighshire  
LL18 2HG

Tel: 01824 706005

E-mail: [cefndy.sales@denbighshire.gov.uk](mailto:cefndy.sales@denbighshire.gov.uk)