

**Information Sheet. Product Code: 4306P**

**Product: Economy Adjustable  
Toilet Seat & Frame**



**User weight limit  
165kg (25 stones)**

**Also available:  
4306F – Adjustable  
height with floor  
fixing legs.**

**4306PL – As 4306P but  
with a seat lid.**

Should you have a problem with  
this device, please quote serial  
number attached to the frame.

**DESCRIPTION:**

A sturdy metal toilet frame, with a hinged black plastic toilet seat, over a removable splash-guard. The seat can be adjusted in height from 47cm to 63cm by gently prising out the pin clips on each leg and sliding the telescopic extension to the desired height.

**INTENDED PURPOSE:**

To be used as a portable toilet seat with an easily disposable toilet facility.

**INSTRUCTIONS FOR USE:**

**Sitting**

Approach the frame, turn around and feel the horizontal bar with the back of your legs.

Put both hands on the handles and lower oneself gently on the toilet seat.

**Standing**

Put both hands on the handles and push up into standing position.

**FITTING INSTRUCTIONS:**

Place in a convenient room and position for client's use. Place a toilet bowl / pot directly under the splash-guard.

Adjust the surround until the handles are at a comfortable height. Do this by gently prising out the pin clips on each leg and sliding the telescopic extension to the desired height.

**SAFETY WARNING:**

Ensure the legs have been adjusted to the same height by counting the number of holes in view.

**Do not stand on the frame.**