



EXPERTS IN PATIENT HANDLING SOLUTIONS



CAR SLIDE SEAT



USER INSTRUCTIONS

The **Car Slide Sheet** is a slide sheet designed to help swivel in and out of a car with ease.

This guide covers all products codes ending with SY.



CAR SLIDE SEAT

If you need a bit of help swivelling in and out of a car, then this **Car Slide Sheet** will be perfect for you. Just tuck it between the back and base of the seat to help secure it, sit back in the seat and swivel around.

It can be used on it's own or in combination with our **Leg Lifter**. This is a device that will make car travel easier.

NOTE: Always remove before driving or while the vehicle is moving.

USES

- Helping people remain independent
- Can also be used with a little assistance from a carer.

FEATURES

- Works well with all types of car seat shapes and designs
- Tucks between the back and base of most car seats to secure
- Washable 40°C to 74°C and low tumble dry
- Easy to insert and remove
- Small and compact for easy storage

BENEFITS

- Offering superb slide, this product enables the majority of people to rotate into/out a car seat with great ease
- Helps people remain independent
- Encourages mobility and rehabilitation
- Makes car travel easier

USED BY

- Independent users
- Carers supporting patients/clients to get into and out of vehicles



Car Slide Seat	Size	Product Code
Car Slide Seat	70cm x 80cm	GS7080SYC
Car Slide Seat	90cm x 90cm	GS9090SYC

Please note: Remove Before Driving!

To view our full range of products visit our **website** at
www.hospitaldirect.co.uk

INSTRUCTIONS



1

Place the **Car Slide Sheet** on the seat to be used by the person requiring some assistance.

This can be used both on the driver or passenger sides but **MUST BE REMOVED PRIOR TO THE CAR MOVING.**

With the open sides running parallel to the seat, tuck the **Car Slide Sheet** between the back rest and the seat where it will be secured.

2



3

The user can then lower themselves onto the seat. They may support themselves by holding onto any part of the car.

4

The user can then gently swivel around so they are facing front. If they need help with their legs, they can use a **Leg Lifter** to assist – either independently or with help from a carer.



5

REMOVING THE CAR SLIDE SHEET

Leaning in towards the middle of the car, the **Car Slide Sheet** can simply be pulled gently out from underneath the user's bottom.

6

The **Car Slide Sheet** can be folded up and placed in the door pocket ready to be used to get out at the end of the journey.



Wash at 40-74°C.
Tumble dry on low heat
Do not iron
Do not use bleach
Do not use fabric conditioner



To view our full range of products visit our **website** at
www.hospitaldirect.co.uk


Assessment Guides

At Hospital Direct we understand that not everyone is familiar with all equipment, so to help therapists and healthcare professionals in assessing a product's suitability for a client, we have included two assessment tools, Functional Independence Measure (FIM) and Easy Guide Symbols. These will enable the Healthcare professional to assess their client and decide whether or not a product will help the person based on their level of dependence and functional ability.

Functional Independence Measure (FIM) Assessment

Our FIM assessment guide for each product will help you decide the suitability of this product for the person's ability and need.

Based on the standard criteria from Level 6 where the person can use the product unaided and unsupervised to Level 1 where all the assistance is provided by the carer and the client can do nothing, this guide is easy, quick and simple to use to check suitability against ability and circumstance.

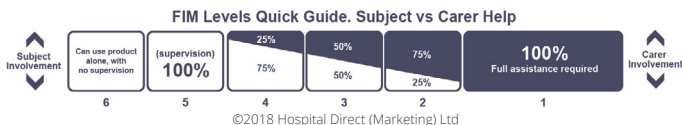

Additional Guide Symbols

We use four symbols throughout our catalogue to indicate where a product is suitable and safe for the person to use unassisted, with a single carer or with multiple carers. These symbols indicate the minimum recommendation.



Person Unassisted



Single Carer



Two Carers



Four Carers

Hospital Direct is a Shropshire based UK manufacturer and supplier of a wide range of Patient Handling Products. With over 25 years experience in the field of patient handling, Hospital Direct is the UK's leading manufacturer and supplier of this type of equipment in the UK – with probably the widest range of any supplier in Europe.

Packaging: To reduce wastage we use recyclable cardboard boxes. If you require different packaging, please call us to discuss.

Disposal: Dispose of in the normal way in accordance with your organisational procedures. If they have been used on highly infected patients please dispose of as such.

Hospital Direct has a policy of continuous development and, as such, reserves the right to alter specifications (including measurements, materials and colours) without prior notice.

UK: Medical Devices Regulations 2002 (SI 2002 No 618, as amended) (UK MDR 2002).

EU: Class I non-invasive devices registered under or complying with the Medical Devices Directive (MDD) 93/42/EEC (as amended in 2007).



Visit our website at
www.hospitaldirect.co.uk

Call: **+44 (0)1588 640144**

For sales orders: sales@hdigroup.co.uk

For information: info@hdigroup.co.uk

Hospital Direct (Marketing) Ltd, Units 3 & 4, The Green, Clun, Shropshire SY7 8LG

VISIT THE WEBSITE

QUOTES IN 3 MINUTES

PRODUCT VIDEOS

SECTOR SPECIFIC PRICING

**FOR A 5%
DISCOUNT VISIT
OUR WEBSITE AND
USE THE CODE
5-PERCENT-OFF**

