

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The portable Transfer Step provides a raised support for feet whilst sitting, transferring from sit to stand or as a step up to the toilet. It is a lightweight, sturdy and slip resistant solution designed to fit in and around obstacles. The front cut out enables the carer to stand as close as possible to the client.

Dimensions:

390 x 320 x 100mm (15 ¼" x 12 ½" x 4")

2.0 Safety Precautions

Do not move step when in use by client.

Avoid using the step on a wet or slippery floor.

Take care to provide client with the necessary support when they are using the step.

3.0 Unpacking

Remove all packaging and check the product is not damaged. Any suspected faults should immediately be reported to the supplier and the product taken out of action.

4.0 Use

Position the step at the feet of the client (or edge of toilet etc.) such that when the client stands, their feet will be shoulder width apart and central upon the step.



5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

To disinfect, wipe clean with a mild disinfectant solution and rinse thoroughly with fresh water.

Please check the suitability of the cleaning products before use. Do not use abrasive cloths or fluids as these could permanently damage the product.

Ensure the step is thoroughly dried before use.

6.0 Maintenance

Periodically check the step for signs of wear or damage. Any suspected faults should be immediately reported to the supplier and the product taken out of service.

7.0 Product Disposal

Product is made from plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

Customers who purchased this product also purchased:



Handling Belts

The NRS range of Handling Belts are easy to use and are comfortable and secure for clients who need extra support.



Support Rails

NRS offer a variety of support rails for use in and around the bathroom. Particularly to aid the sit and rise from the toilet.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk