

Neater Steady-Double-Straw (ND-KV2) - Instructions

Description

- **Neater Steady-Double-Straw (ND-KV2)** provides a stable but adjustable holder for two flexible straws. (Eg: making a cup of tea accessible now and water later)
- The non-return valves hold the liquid in the straws making sucking easier.

Set up instructions - See picture below.

- Clamp to table.
- Clip in flexible tubing.
- Bend to a convenient position for the user.
AVOID SHARP BENDS and DO NOT pull hard on the bendy arm as this can damage it.
- Use Velcro strap to tidy set up.
- Take care that the cup used is stable enough and will not be pulled over by the straw.
- Some people may benefit from a short length of the rigid tubing being pushed all the way inside the mouthpiece end of the flexible tube.


Using the Cleaning Brush

- Feed the brush handle end without the bristles all the way into the tubing and pull it through.
- Repeat as necessary, pushing backwards and forwards.
- Food safe cleaning and/or sterilising solution can be used.
- Rinse

Safety



- Drinkers should only be used for drinking consumable liquids.
- They should not be used by people with swallowing difficulties or who are liable to choke, except under the supervision of a trained Speech and Language Therapist.
- Drinks should not be scalding hot.
- Your mouth should always be higher than the mug or siphoning can occur and cause choking.
- Clean all parts before use and check that no cracks have appeared in any components or that any parts can come loose and cause choking.
- Take care not to stick the straw in your eye or into the back of your throat.

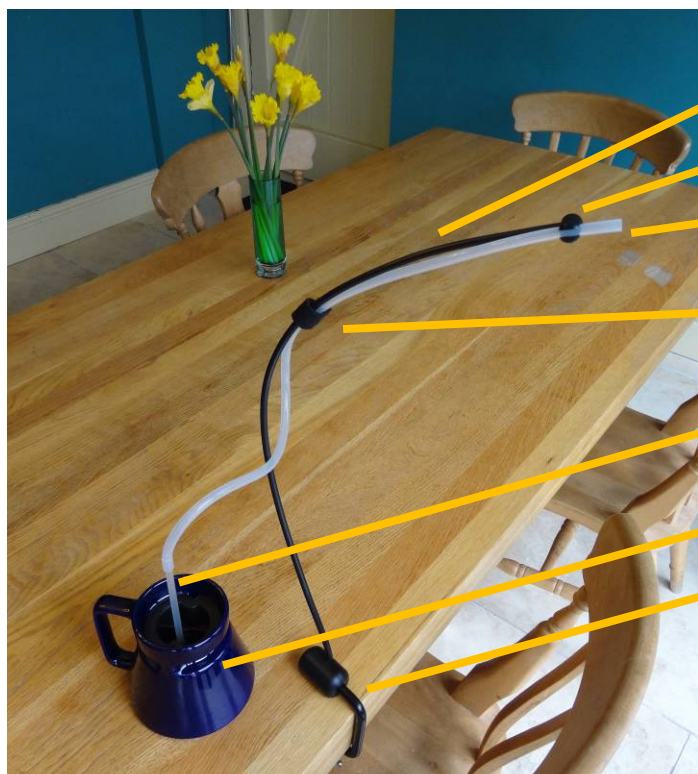

Bendy Arm

Double Clip

Flexible Tubing (one shown)

Velcro Strap

Short rigid straw and non-return valve

Use a sturdy cup (not included)

Clamp

