

Instructions for the safe fitting and use of Aquarian Shower and Bathboards



Weight Limit 190kg (30st)

Code ASB6226 230x660mm

Code ASB6227 230x685mm


Code ASB6228 230x710mm

Code QUSB6626 350x660mm

Code QUSB6627 350x685mm

Code QUSB6628 350x710mm

Manufactured in Taiwan for SmitCare

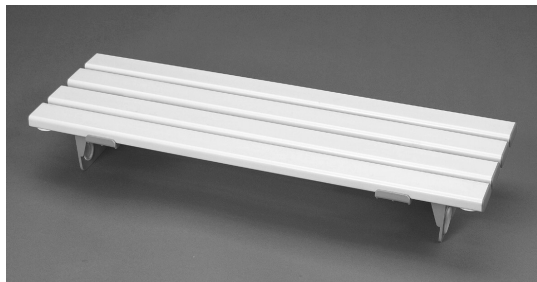


Important Notes

A risk assessment must be carried out as to the suitability of this product particularly with new users. The following risks should be considered: Is the bath suitable for this board? Is the board weight appropriate? Is it the correct size for the bath? Will user's health require further regular reviews?

Make sure your Aquarian Bathboard is the correct length for the width of your bath. To avoid the risk of tipping make sure your bathboard does not overhang the edge of the bath by more than 20mm. Always make sure that your Aquarian Bathboard is fixed securely before use. It is particularly important that there is no lateral movement. Position as close to the end of the bath as possible – where the bath is straight, not tapered.

- 1) The grommets on the cleats must face outwards towards the sides of the bath to ensure a firm fitting; this applies to both cleats. It may be necessary to reverse the direction of the cleats through 180° because of the way the bathboards are packed.
- 2) Loosen the wing screws and move the cleats towards the centre of the board for the narrowest fitting.
- 3) Place the board across the bath at the position where it is to be used, with the cleats on the underside. The rim of the bath should support at least 40mm (1.5") of the end of the board, on both sides of the bath.
- 4) Reach under the board and push the cleats outwards until they touch the inside walls of the bath, then tighten the wingscrews.
- 5) Remove the board from the bath, turn it over and rest it on a flat surface. Loosen one of the wingscrews sufficiently to allow the cleat to be moved out by one notch, then re-tighten the wingscrew. Replace the board on the bath the correct way up.
- 6) Your bathboard should be firm with no lateral movement. If there is any movement, repeat step 5 until you are satisfied with the fitting.

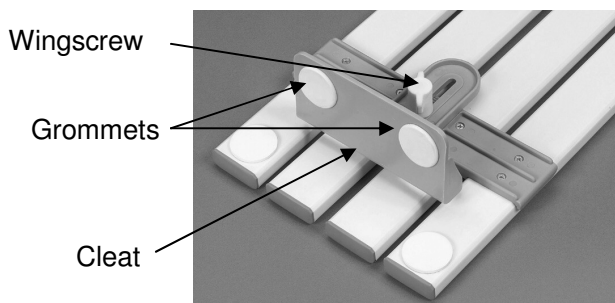

Note: The board should be checked for safe fitting each time it is used. Use the board as directed by your Occupational Therapist.

Cleaning: May be steam cleaned or Autoclaved for 3 minutes at 60°C.
 Bathrooms can be a source of infections, therefore clean regularly with a solution of mild detergent and household disinfectant in warm water applied with a damp sponge.
 Dry thoroughly. Do not clean with harsh abrasives.

Warranted to specification for one year

Questions / comments, please contact our
 customer services department on Lo-call : **0845 1303 747**