

Information Sheet. Product Code:4550 / 4553 Product: Mobile / Fixed Height Shower Chair

4550 Illustrated



**Maximum user weight –
140kg (22 stones)**

Should you have a problem with this device, please quote the batch number attached to the frame.

DESCRIPTION:

A portable shower chair, which allows the user to sit in comfort whilst taking a shower. The seat is perforated with small drainage holes to give effective drainage but remains safe for male use. The rubber ferrules on the feet will reduce slippage and marks on the shower floor.

Width between arms – 47cms (18 ½")

4550 – Braked castors

4553 – Standard feet (rubber ferrules)

INSTRUCTIONS FOR USE:

No special instructions or training is required. The 4550 model is equipped with castors for ease of mobility, the front two can be braked when in use. To operate simply press the two rear tabs down on the castors. To release the brakes push down on the grey plastic tabs.

Always keep the castors clear from debris such as hair etc. which may lead to the castors sticking.

SAFETY WARNING:

The 4550 model is not intended to be used as a wheelchair. It is designed to move persons in / out of a shower area to their bespoke mobility aid.

Ensure that where genital stoppers are used, they are in position.

Only use on level surfaces. Should the ferrules or castors become damaged discontinue use and seek replacements.

This device is not suitable for standing on or to be submerged in pools etc.