

User Instructions for Toilet Surrounds

T76 – T77 – T78 – T76R – X154 – X155

Introduction

Congratulations on your purchase of your Cefndy Healthcare Toilet Surround. Please ensure you read and understand these instructions in full before construction and use. Please retain these for future reference.

This Toilet Surround gives support and confidence to sit and rise – it offers support and lessening of strain. The height is adjustable to cater for varying requirements.

The Cefndy toilet surrounds are available with a variety of features. They are available with/without padded arms and with free standing or floor fixing feet. In addition the T76 is available in Red as part of our dementia range. Toilet surrounds are also available in bariatric sizes.

	T76 / T76R	T77	T78	X154	X155
Width between Arms	490mm 19¼"	490mm 19¼"	490mm 19¼"	550mm 21¾"	550mm 21¾"
Floor to top of Arms	630 – 780mm 24¾" – 30¾"	640 – 790mm 25" – 31"	630 – 780mm 24¾" – 30¾"	650mm 25½"	650mm 25½"
Footprint	520 x 555mm 20½" – 21¾"	520 x 555mm 20½" – 21¾"	520 x 555mm 20½" – 21¾"	615 x 490mm 24¼" x 19"	615 x 490mm 24¼" x 19"
Maximum User Weight	159kg 25st	159kg 25st	159kg 25st	254kg 40st	254kg 40st



Pack Contents

- Toilet Surround
- Legs
- Pin clips
- User Instructions

Assembly Instructions

Please note:

- When unpacking carefully examine the product to ensure it is complete.
- Before use check for any damage. If damaged please contact your supplier and refrain from using the product.
- **Do not attempt to use if the product is damaged.**

Correct Use

Height Adjustment

Remove the pin clip from each of the legs.

- Select the required height of the frame by gently sliding each extension leg in or out.
- When desired height is found refit the pin clip into the appropriate hole in each extension leg.
- Ensuring it goes fully through. Ensure that all legs are set to the same height.

Cleaning

- This product can be cleaned using a mild detergent and a mild disinfectant or antibacterial solution. The framework should be wiped down on at least a weekly basis. Caution must be taken to understand the suitability of cleaning agents/chemicals being used.
- Do not use an abrasive cleaner as this may damage the frame and invalidate your warranty.
- It is recommended that the potty is thoroughly cleaned after every use, using an appropriate cleaning agent.

Use of power/jet washing & industrial washing machines

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependent on the manufacturer's instructions.

Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

Warning

- The product is intended for users of a maximum weight as detailed above.
- We recommend a full risk assessment is carried out before use. This should include consideration for the equipment's intended use, the capacity of the user to safely use the equipment, the environment in which the equipment is intended to be used and **all** other relevant considerations.

Warranty

This product carries a 5 year warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

Disposal Instructions

Appropriate hygiene standards should be applied before disposal to minimise the risk of infection.

- Materials used are essentially non-hazardous and can be disposed of appropriately consideration should be given to local re-cycling arrangements.

Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 01824 706005
E-mail: cefndy.sales@denbighshire.gov.uk