Etac Turner

Safe transfer and gentle to your back



The importance of safe transfers

Persons in need of assistance often feel unsafe in transfer situations, and caregivers are subject to strain when assisting with the transfer. The risk of accidents and insecurity from both caregiver and the assisted person may result in fewer transfers. Being left in the same position for long term can lead to severe secondary injuries. With Turner, both the assisted person and the caregiver feel safe and the transfer is performed with ease. Turner is the perfect tool for both nursing homes and home care.

Easy to use for both caregiver and care recipient

Turner is small and handy and can easily be placed close to the user. The low footplate has two large anti slip foot prints to facilitate proper position of the feet onto the base. The lower leg support shall be set in height just under the care recipient's knee and the handle slightly above the shoulder level. Now Turner is ready to use. The caregiver puts one foot on the pedal, leans backwards in order to counterweight while the user grips the handle, presses the lower legs towards the support and stands up. A safe transfer which allow eye contact and good communication.



The caregiver takes hold of the upper handle with both hands, puts one foot on the pedal and leans slightly backwards to act as counterweight.



Two models

Turner comes in two models, with Oval or U-handle.



The assisted person grips the lower handle, presses the lower legs toward the support and stands up.



Padded lower leg support

A wide and soft padded support with a slightly curved design. It gives a secure and stable support. Stepless height adjustment. A measuring scale indicates the adjusted height.



Anti-slide protection

For correct and secure positioning of the feet.



Turner is rotated to place the assisted person correctly in front of the new seat. Use the same technique as for standing up. While the assisted person is in a fully standing position, the caregiver doesn't need to keep the foot on the pedal.



Transport wheels

The wheels are only intended for transportation of Turner without user.

DescriptionTurner Oval-handle Turner U-handle

Item no. 16090104 16090105

Weight

Max user weight: 150 kg/330 lbs Total weight: 12,8 kg/28,2 lbs

Dimensions

U-handle height: 82-123 cm/33"-48.4" Oval-handle height: 78-125 cm/30.7"-49.2" Footplate diameter: 41 cm/16" Footplate height: 13 mm/1/2" Lower leg support width: 41 cm/6" Lower leg support height: 15 cm/6" Material: Powder coated steel

Colour White

Compliant with MDD 93/42/EEC EN 12182



85°c 185°





Clean the product with a solvent-free cleaning agent (pH level 6–9), or with a 70% disinfectant solution. Can be decontaminated at max 85° C / 185° F if so required.

Etac AB
Kista Science Tower
SE-164 51 Kista, Sweden
Tel +46-371-58 73 30
Fax +46-371-58 73 90
info@etac.se www.etac.com

